

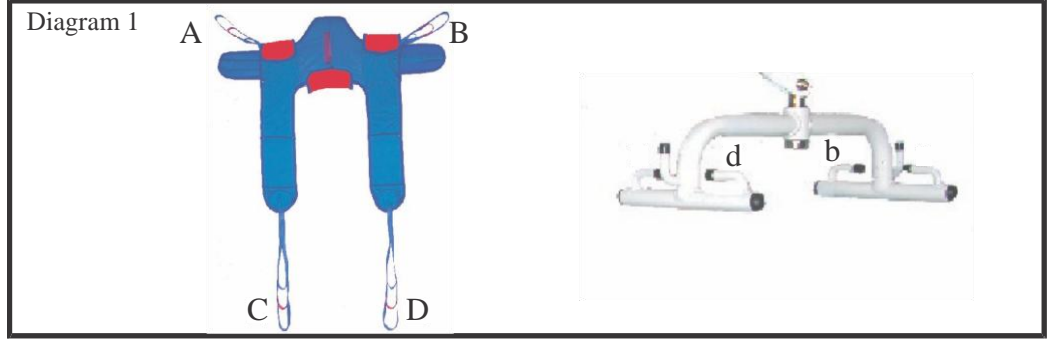


# Bestcare Sling User Guide

## SANI SLINGS (Toileting)

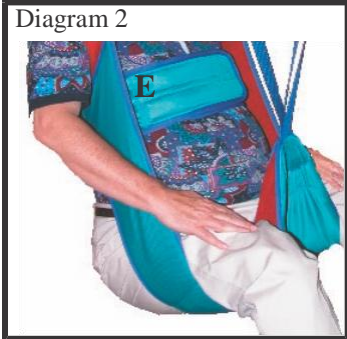

### WARNING!

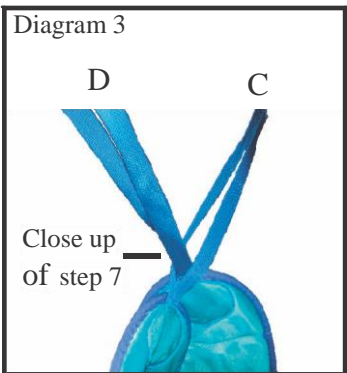

Step 1: Check sling weight capacity for the patient being lifted.

Step 2: Review following procedures carefully prior to attempting lift. Call with any questions.

Step 3: Position the sling under and behind the back with color adjustment strap outside. The two red padded sections next to A & B should be positioned under the arms (Diagram 2).



Step 4: Cross the Velcro belt (E) over chest and fasten as in Diagram 2.

Step 5: Loop straps C & D around and under legs as in Diagram 2.

Step 6: Hook loops A & B to spreader bar a & b with arms outside the sling loops (Diagram 4).

Step 7: Cross over C & D placing loop C through loop D as in Diagram 3.

Step 8: Hook loops C & D to hooks c & d on spreader bar as in Diagram 4.

Step 9: Do not lock caster brakes to allow lift to properly balance weight.

Step 10: Proceed with lift and transfer to commode as in Diagram 5.



1-877-822-9033