


## Washing Instruction Symbols and Definition

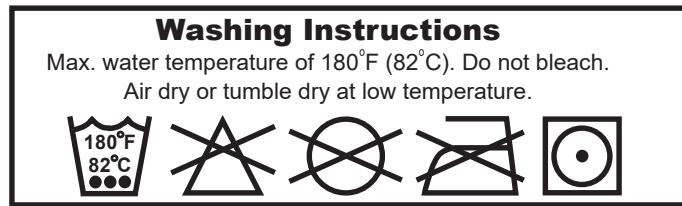

1. Machine Wash, Hot. Initial water temperature should not exceed 82°C or 180°F.
2. Do Not Bleach. No bleach product may be used. The garment is not colorfast or structurally able to withstand any bleach.
3. Do Not Dry Clean. Garment may not be commercially dry cleaned.
4. Do Not Iron. Item may not be smoothed or finished with an iron.
5. Tumble Dry, Normal, Low Heat. A machine dryer may be regularly used at a maximum of Low Heat setting.

## Warranty

Each Bestcare sling is manufactured and tested using approved industry standards. Bestcare LLC warrants that all Bestcare slings are free from defects in materials and workmanship for a period of six months from the date of purchase. It is the responsibility of the purchaser to verify the integrity of each sling prior to each and every use. Slings that have been altered or repaired by the purchaser in any way or washed in an unsafe manner are not covered under warranty.


# Invacare Replacement Sling Owner's Manual

**Bestcare**  
L.L.C.

2920 Pacific Dr  
Norcross, GA 30071  
678-679-6690

## Invacare Full Body w/ Commode

## Invacare Full Body

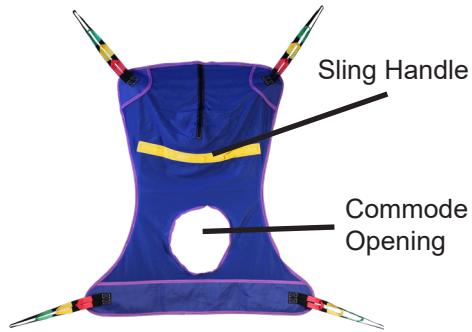

Model	Description	Size	Max Weight	Color Code
SL-R110	Full Body Mesh	M	450 lbs / 205 kg	Yellow
SL-R111	Full Body Mesh	L	450 lbs / 205 kg	Green
SL-R112	Full Body Solid	M	450 lbs / 205 kg	Yellow
SL-R113	Full Body Solid	L	450 lbs / 205 kg	Green
SL-R114	Full Body Mesh w/Commode	M	450 lbs / 205 kg	Yellow
SL-R115	Full Body Mesh w/Commode	L	450 lbs / 205 kg	Green
SL-R116	Full Body Mesh w/Commode	X-L	450 lbs / 205 kg	Blue
SL-R117	Full Body Solid	X-L	450 lbs / 205 kg	Blue
SL-R140	Full Body Mesh	XX-L	600 lbs / 273 kg	Black
SL-R141	Full Body Mesh w/Commode	XX-L	600 lbs / 273 kg	Black



Refer to Owner's Manual for more operating instruction.



Warning!

This sling is **ONLY** compatible with 6 pt or 2 pt Bestcare spreader bars

DO NOT exceed the sling weight capacity.

**IF** any slings with higher load capacity than the equipment itself is used, **DO NOT** load the equipment more than the lowest maximum safe working load.

Do not alter the slings.


If the sling is frayed, torn, ripped, loops broken or any stitching is not secured, it is not safe to use the sling and must be replaced immediately!

Prior to lifting also ensure the size of the sling is appropriate for the person being lifted. Using a sling too small or too large is very dangerous and may result in serious injury.


Do not use the slings if you or the staff is not familiar with these procedures. Bestcare LLC accepts no liability for slings used incorrectly or after they have been damaged or are in need of replacement.

Please contact Bestcare LLC or your Bestcare dealer if you have any questions regarding the safe usage of Bestcare slings.


# Sling Hook-Up Guide

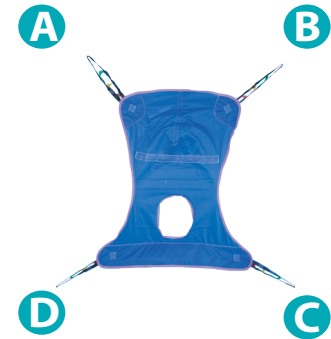

Hook up A ➡ Point 1  
Hook up B ➡ Point 2  
Hook up C ➡ Point 3  
Hook up D ➡ Point 4

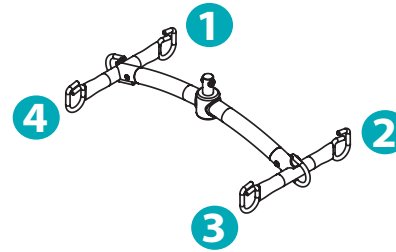

Any 6 points spreader bar

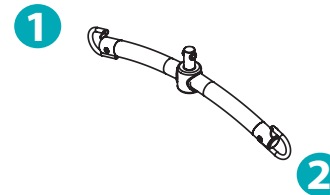

2 points spreader bar

Hook up A ➡ Point 1  
Hook up B ➡ Point 2  
Hook up C ➡ Point 2  
Hook up D ➡ Point 1

## Transfer from wheelchair



Scan code for video demonstration

1. Slide the sling down between the seat and the user's back. Use the sling handle on the back of the sling to help pushing the sling down to the lower back.
2. Pull the leg sections (C and D) out from under the thighs until the sling reaches to the knee
3. Follow the guide above to hook the sling to the spreader bar.