


## Washing Instruction Symbols and Definition

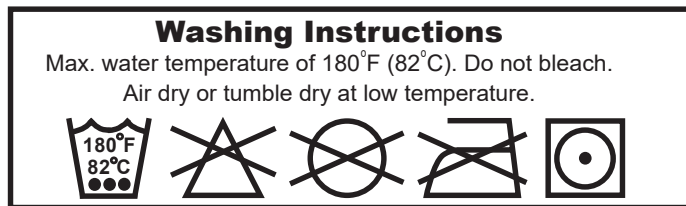

1. Machine Wash, Hot. Initial water temperature should not exceed 82°C or 180°F.
2. Do Not Bleach. No bleach product may be used. The garment is not colorfast or structurally able to withstand any bleach.
3. Do Not Dry Clean. Garment may not be commercially dry cleaned.
4. Do Not Iron. Item may not be smoothed or finished with an iron.
5. Tumble Dry, Normal, Low Heat. A machine dryer may be regularly used at a maximum of Low Heat setting.

## Warranty


Each Bestcare sling is manufactured and tested using approved industry standards. Bestcare LLC warrants that all Bestcare slings are free from defects in materials and workmanship for a period of six months from the date of purchase. It is the responsibility of the purchaser to verify the integrity of each sling prior to each and every use. Slings that have been altered or repaired by the purchaser in any way or washed in an unsafe manner are not covered under warranty.

## Pediatric Sling Owner's Manual

**Bestcare**  
L.L.C.

2920 Pacific Dr  
Norcross, GA 30071  
678-679-6690

Pediatric Hammock Padded

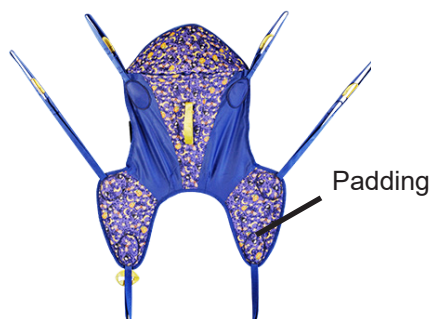

Pediatric Hammock Mesh



Model	Description	Size	Max Weight	Color Code
SL-PM111	Pediatric Hammock Mesh	Child	150 lbs / 68 kg	Yellow
SL-PP121	Pediatric Hammock Padded	Child	150 lbs / 68 kg	Yellow



Refer to Owner's Manual for more operating instruction.



Warning!

This sling is **ONLY** compatible with 6 pt or 2 pt Bestcare spreader bars

DO NOT exceed the sling weight capacity.

**IF** any slings with higher load capacity than the equipment itself is used, **DO NOT** load the equipment more than the lowest maximum safe working load.

Do not alter the slings.



If the sling is frayed, torn, ripped, loops broken or any stitching is not secured, it is not safe to use the sling and must be replaced immediately!

Prior to lifting also ensure the size of the sling is appropriate for the person being lifted. Using a sling too small or too large is very dangerous and may result in serious injury.

Do not use the slings if you or the staff is not familiar with these procedures. Bestcare LLC accepts no liability for slings used incorrectly or after they have been damaged or are in need of replacement.


Please contact Bestcare LLC or your Bestcare dealer if you have any questions regarding the safe usage of Bestcare slings.

# Sling Hook-Up Guide

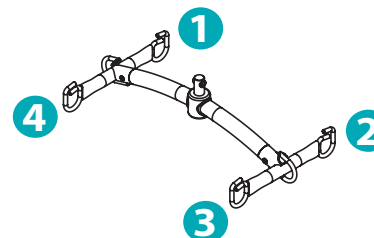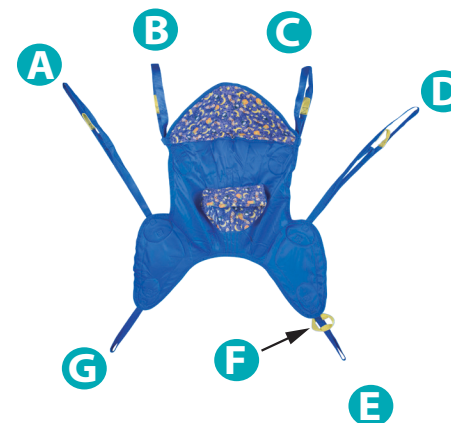

Any 6 points spreader bar

- Hook up B ➡ Point 1
- Hook up C ➡ Point 2
- Insert G through F
- Insert A through G
- Insert D through E
- Hook up A ➡ Point 4
- Hook up D ➡ Point 3

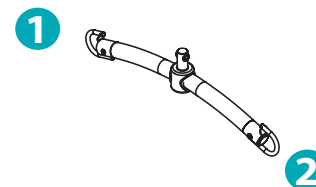

2 points spreader bar

- Hook up B ➡ Point 1
- Hook up C ➡ Point 2
- Insert G through F
- Insert A through G
- Insert D through E
- Hook up A ➡ Point 1
- Hook up D ➡ Point 2

## Transfer from wheelchair

1. Slide the sling down between the seat and the user's back. Use the sling handle on the back of the sling to help pushing the sling down to the lower back.
2. Secure the patient using the waist belt.
3. Feed the leg sections (E, F and G) under the thighs.
4. Gently pull the leg section up the inner thigh.
5. Follow the guide above to hook the sling to the spreader bar.