


Washing Instruction Symbols and Definition

Washing Instructions

Max. water temperature of 180°F (82°C). Do not bleach.

Air dry or tumble dry at low temperature.

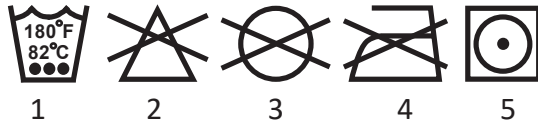

1. Machine Wash, Hot. Initial water temperature should not exceed 82°C or 180°F.
2. Do Not Bleach. No bleach product may be used. The garment is not colorfast or structurally able to withstand any bleach.
3. Do Not Dry Clean. Garment may not be commercially dry cleaned.
4. Do Not Iron. Item may not be smoothed or finished with an iron.
5. Tumble Dry, Normal, Low Heat. A machine dryer may be regularly used at a maximum of Low Heat setting.

Warranty

Each Bestcare sling is manufactured and tested using approved industry standards. Bestcare LLC warrants that all Bestcare slings are free from defects in materials and workmanship for a period of six months from the date of purchase. It is the responsibility of the purchaser to verify the integrity of each sling prior to each and every use. Slings that have been altered or repaired by the purchaser in any way or washed in an unsafe manner are not covered under warranty.

Stand Assist Sling Owner's Manual

Bestcare
L.L.C.

2920 Pacific Dr
Norcross, GA 30071
678-679-6690

Stand Assist Sling



Model	Description	Size	Max Weight	Color Code
SL-SA661	Stand Assist Deluxe Padded	S	400 lbs / 182 kg	Red
SL-SA662	Stand Assist Deluxe Padded	M	400 lbs / 182 kg	Yellow
SL-SA663	Stand Assist Deluxe Padded	L	450 lbs / 205 kg	Green
SL-SA664	Stand Assist Deluxe Padded	XX-L	600 lbs / 273 kg	Black
SL-SA669	Stand Assist Buttock Strap	Standard	400 lbs / 182 kg	Yellow
SL-SA69B	Stand Assist Buttock Strap	XX-L	600 lbs / 273 kg	Black



Refer to Owner's Manual for more operating instruction.



Warning!

This sling is only compatible with Bestcare Stand Assist Lift

DO NOT exceed the sling weight capacity.

IF any slings with higher load capacity than the equipment itself is used, **DO NOT** load the equipment more than the lowest maximum safe working load.

Do not alter the slings.

If the sling is frayed, torn, ripped, loops broken or any stitching is not secured, it is not safe to use the sling and must be replaced immediately!


Prior to lifting also ensure the size of the sling is appropriate for the person being lifted. Using a sling too small or too large is very dangerous and may result in serious injury.

Do not use the slings if you or the staff is not familiar with these procedures. Bestcare LLC accepts no liability for slings used incorrectly or after they have been damaged or are in need of replacement.


Please contact Bestcare LLC or your Bestcare dealer if you have any questions regarding the safe usage of Bestcare slings.

Sling Hook-Up Guide


Note: **A** has a blue strap and a black safety strap!

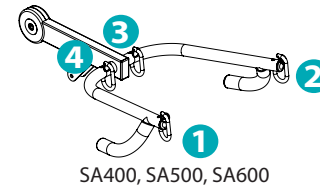

SA400, SA500, SA600

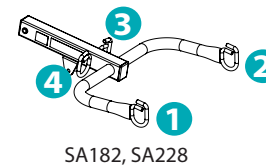

SA182, SA228

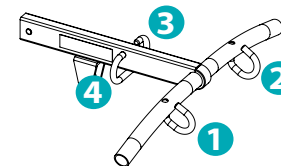

SA400H, SA400HE

Insert B through Black strap of A
Hook up B ➡ Point 1
Hook up Blue Strap of A ➡ Point 2

Transfer from wheelchair

1. Slide the sling down between the seat and the user's back. Use the handle on the back of the sling to help pushing the sling down to the lower back.
2. Wrap the waist belt around the user's waist
3. Buckle the belt buckle around the waist and adjust the strap to secure the waist belt tight around the user's waist.
4. Follow the guide above to hook the sling to the spreader bar.



Scan code for video demonstration