

Door Connectors

Transition gate

Through the door – quick and easy



Primary application

Home care
Nursing homes
Hospital

Door connectors are used for patient transfer from room to room. This simple and reliable mechanism is used for transferring patients from room to room, e.g. bedroom to bathroom, thereby reducing the number of total lifts required. Fewer lifts creates a better working environment for the carer and safer transfer for the patient.

The door connector, as a wall mounted solution, is the preferred choice by architects and building contractors, because of its elegant and compact design. The door connector is suitable for most wall widths. And because it is mounted very close to the wall surface taking up little space in the room, the desired room covering area is exploited optimally.

This door connector is only 11.8" wide and does not require costly changes to the building's electrical system, as it operated manually.

Locking and unlocking the door connector is simply done by lightly pulling on the cord. When the door connector is unlocked and the traverse is aligned with the mechanism, the system locks automatically, ready for room to room transfer. To unlock pull on the cord.

Key standard functions

Compact and discreet
Only 11.8" wide
Flexible for fixed and traverse systems
Manual operation



Bestcare
L.L.C.

ergolet

2920 Pacific Drive
Norcross, GA 30071
U.S.A

www.bestcarellc.com
info@bestcarellc.com



1. Pull the cord to activate door connector. Slide the traverse towards the door connector.



2. When the traverse is aligned with the door connector, it will automatically lock into position with a click sound ready for room to room transfer.




3. When the ceiling hoist has been moved into the other room, pull on the cord to unlock the mechanism and to move the traverse.



Traverse track 5.3"
Connector 5.3"
Traverse track 5.3"



Fixed 5.3"
Traverse track 5.3"

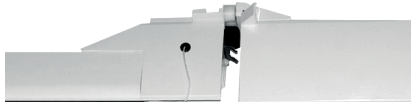

Fixed track 3"
Traverse track 5.3"

PRODUCT HIGHLIGHTS

- Manual operation – no costly electrical installations required
- Traverse is locked and unlocked by simply pulling the cord
- Compact and discreet
- Several models to suit individual configurations
- Fail safe mechanical features


Version	Item-number
Traverse track 5.3" - Connector 5.3" - Traverse track 5.3"	TK-DC-135T/135T
Fixed track 5.3" - Traverse track 5.3"	TK-DC-135F/135T
Fixed track 3" - Traverse track 3"	TK-DC-75F/75T




Unlocking the door connector happens easily by pulling the cord. Pull the cord again to lock the mechanism.

TECHNICAL SPECIFICATION

	Connector 5.3"
Length	11.8"
Height	5.7"
Width	3.9"

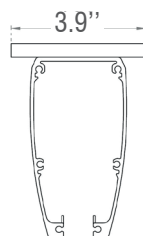


Drawings refer to connector 5.3" mm

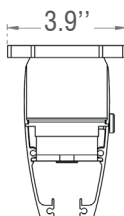

Drawings refer to fixed track 3" mm

	Fixed track 3"
Length	8.7"
Height	5.7"
Width	3.9"

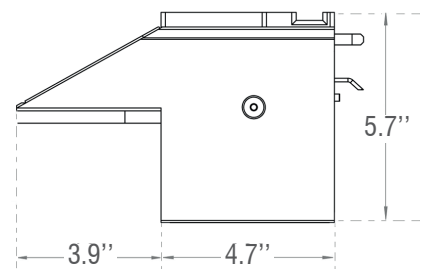

Consult the Technical manual for further specifications. All figures and measurements are for guidance only and may differ in actual situations. Bestcare reserves the right to make changes without notice.

Product photos may include accessories. Consult your dealer.



2920 Pacific Drive
Norcross, GA 30071
U.S.A
www.bestcarellc.com
info@bestcarellc.com