


## Washing Instruction Symbols and Definition

### Washing Instructions

Max. water temperature of 180°F (82°C). Do not bleach.

Air dry or tumble dry at low temperature.

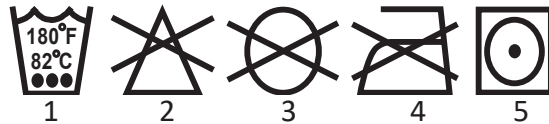

1. Machine Wash, Hot. Initial water temperature should not exceed 82°C or 180°F.
2. Do Not Bleach. No bleach product may be used. The garment is not colorfast or structurally able to withstand any bleach.
3. Do Not Dry Clean. Garment may not be commercially dry cleaned.
4. Do Not Iron. Item may not be smoothed or finished with an iron.
5. Tumble Dry, Normal, Low Heat. A machine dryer may be regularly used at a maximum of Low Heat setting.

### Disposable Slings



**Warning! DO NOT WASH disposable slings!!**

**If label is red  
do not use**

If this label is red, the disposable sling is NOT SAFE to use. Replace immediately!

### Warranty


Each Bestcare sling is manufactured and tested using approved industry standards. Bestcare LLC warrants that all Bestcare slings are free from defects in materials and workmanship for a period of six months from the date of purchase. It is the responsibility of the purchaser to verify the integrity of each sling prior to each and every use. Slings that have been altered or repaired by the purchaser in any way or washed in an unsafe manner are not covered under warranty.

## Guldmann/Liko Replacement Sling Owner's Manual

**Bestcare**  
L.L.C.

2920 Pacific Dr  
Norcross, GA 30071  
678-679-6690

## Guldmann/Liko Replacement Sling

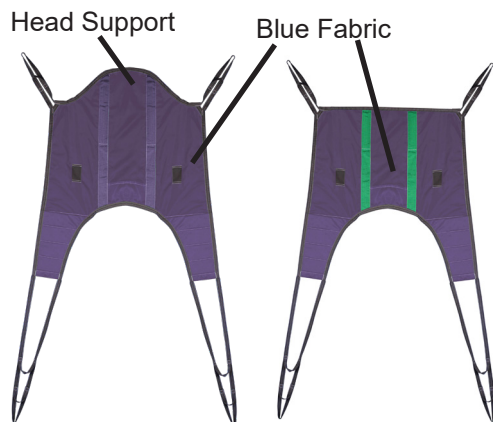

MODEL	DESCRIPTION	SIZE	MAX. WEIGHT
SL-GP472	Universal Padded	M	600 lbs
SL-GP473	Universal Padded	L	600 lbs
SL-GP474	Universal Padded	XL	600 lbs
SL-GPH482	Universal Padded w/ Head Support	M	600 lbs
SL-GPH483	Universal Padded w/ Head Support	L	600 lbs
SL-GPH484	Universal Padded w/ Head Support	XL	600 lbs



Refer to Owner's Manual for more operating instruction.



Warning!

This sling is only compatible with 6 pt or 2 pt Bestcare spreader bars

DO NOT exceed the sling weight capacity.

**IF** any slings with higher load capacity than the equipment itself is used, **DO NOT** load the equipment more than the lowest maximum safe working load.

Do not alter the slings.


If the sling is frayed, torn, ripped, loops broken or any stitching is not secured, it is not safe to use the sling and must be replaced immediately!

Prior to lifting also ensure the size of the sling is appropriate for the person being lifted. Using a sling too small or too large is very dangerous and may result in serious injury.

Do not use the slings if you or the staff is not familiar with these procedures. Bestcare LLC accepts no liability for slings used incorrectly or after they have been damaged or are in need of replacement.


Please contact Bestcare LLC or your Bestcare dealer if you have any questions regarding the safe usage of Bestcare slings.

## Guldmann/Liko Replacement Disposable Sling

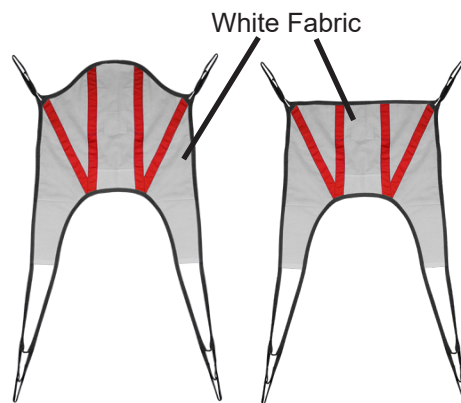


Model	Description	Size	Max Weight
SL-GD432	Universal SPS Padded	M	600 lbs
SL-GD433	Universal SPS Padded	L	600 lbs
SL-GD434	Universal SPS Padded	XL	600 lbs
SL-GDH442	Universal SPS Padded w/Head Support	M	600 lbs
SL-GDH443	Universal SPS Padded w/Head Support	L	600 lbs
SL-GDH444	Universal SPS Padded w/Head Support	XL	600 lbs


# Sling Hook-Up Guide

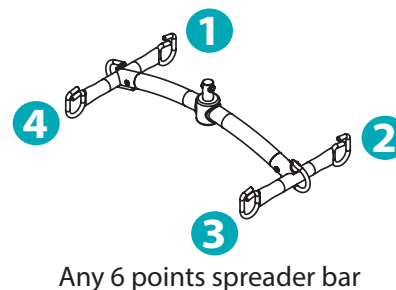

Any 6 points spreader bar

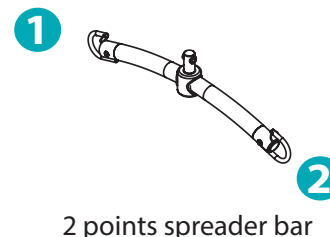

2 points spreader bar

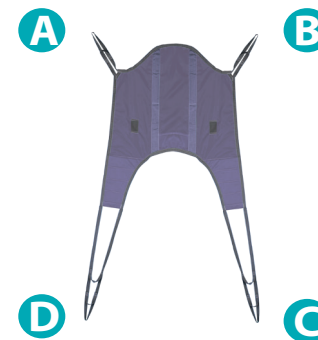

Hook up A ➡ Point 1  
Hook up B ➡ Point 2  
Insert C through D  
Hook up C ➡ Point 4  
Hook up D ➡ Point 3

Hook up A ➡ Point 1  
Hook up B ➡ Point 2  
Insert C through D  
Hook up C ➡ Point 1  
Hook up D ➡ Point 2

## Transfer from wheelchair



Scan code for video demonstration

1. Slide the sling down between the seat and the user's back. Use the pouch on the back of the sling to help pushing the sling down to the lower back.
2. Feed the leg sections (C and D) under the thighs.
3. Gently pull the leg section up the inner thigh.
4. Follow the guide above to hook the sling to the spreader bar.