

# **BESTLIFT®**


FULL BODY PATIENT LIFT

Model: PL400EF

Foldable / Portable

# **Bestcare**

L.L.C.

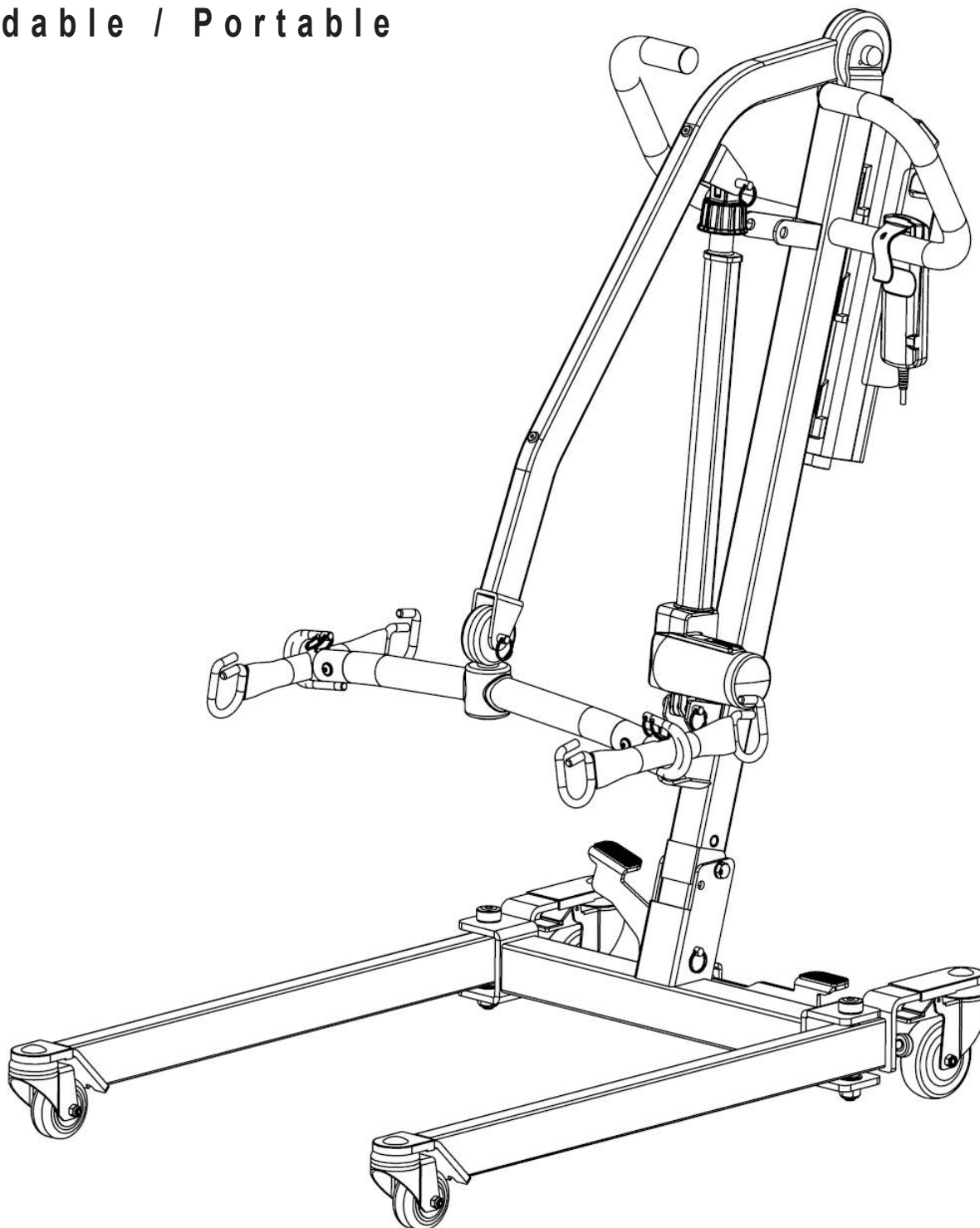

## BEFORE USE:



Ensure the product has been assembled according to the instructions in this manual. All operators should read and understood the instructions for safe and proper operation of the patient lift.



Scan the QR code or use the link to register your product warranty online for instant protection. Registration information can be found on the product label.

<https://www.bestcarellc.com/warranty-registration/>

# Thank you for choosing Bestcare!

To better serve you, please record the following information for future use:

Supplier Name: \_\_\_\_\_

Supplier Telephone: \_\_\_\_\_

Product Serial Number: \_\_\_\_\_

Date of Purchase: \_\_\_\_\_

## Date of Manufacture



The manufacturing date can be found on the serial number label that is located at the top of the mast. The YY in the serial number is the manufacture year, and the MM is the manufacture month.



800519

This product has been tested by SGS, an accredited testing institute to comply with the requirements of:

ISO 10535, IEC 60601-1, ANSI/AAMI ES60601-1, CAN/CSA-C22.2 No.60601-1, IEC 60601-1-11, ANSI/AAMI HA60601-1-11, CAN/CSA-C22.2 No.60601-1-11, IEC 60601-1-2

## **Product Description**

The Bestlift PL400EF allows a patient up to 400 Lb to be lifted and transferred safely with minimal physical effort provided by the operator. This is due to it being a full electric lift . Patients can be lifted from floor to bed with ease. It uses a six point spreader bar. As an added feature, this lift is able to be folded and stored away for people with limited space at home or in the trunk of your car for individuals who like to travel. With the addition of the optional battery pack, you will never have to be worried about your lift not being charged and ready for use.





















## **Intended Use**

This lift is intended to be used with patients who are completely immobile or totally dependent on the patient lift for transfers. It is intended to be used for patients with limited space at home or for customers who like to travel as it folds and fits in the trunk of your car. The lift is intended to be used for transfers of the patient in and out of bed, their wheelchair, to and from the commode, or any other type of surface. This lift is intended to be operated by one caregiver who will perform the lift and transfer. Transfers should always be done on a flat dry surface, and are not intended to be used in high humidity environments for extended amounts of time.

# DEFINITIONS & SYMBOLS

In this manual the user refers to the patient or resident and may be used interchangeably at different times. Caregiver refers to the operator or person who is assisting with the transfer.

Symbols used in this manual and on the product and their meanings:

	Warning! Failure to heed this warning may result in damage to the product or serious injury to the operator and/or user.		Do Not Bleach. No bleach product may be used. The garment is not colorfast or structurally able to withstand any bleach.
	Important instructions follow. Read and understand the instructions in the manual before using the product.		Do Not Dryclean. Garment may not be commercially drycleaned.
	Type BF applied part. Refer to the part of the medical device which come into physical contact with the patient in order for the device to carry out its intended use.		Do Not Iron. Item may not be smoothed or finished with an iron.
	Class II symbol. To identify equipment is designed that a connection to electrical earth (ground) is not required.		Tumble Dry, Normal, Low Heat. A machine dryer may be regularly used at a maximum of Low Heat setting.
	Note! Important information concerning the product and/or its correct and proper usage follows.		Do Not Dry. A machine dryer may not be used.
	Advise the operator to consult the accompanying documents before use.		Machine Wash, Hot. Initial water temperature should not exceed 82°C or 180°F.
	Right side up		Do Not Wash
	Fragile, handle with care.		Keep dry
	Carton storage temperature limit of -25°C to 70°C.		Carton storage humidity limit of 0% to 90%.
	European Conformity mark, the product conforms to European Medical Directive 93/42/EEC and meets applicable health, safety and environmental requirements.		Handle with care

# **Safety Instructions**

This lift will provide years of service if it is properly maintained as any electric and/or mechanical equipment requires. Please pay careful attention to the information in this manual regarding the care, maintenance, and operation of the patient lift. Carefully read these instructions before assembling the lift, or attempting to lift a user with the device.

- Before attempting to lift a patient, please practice operating the lift and explain the lift procedures to the patient
- Please provide special assistance under the caretakers' discretion. (i.e. patients who are comatose, spastic, agitated, or otherwise severely handicapped.)
- The patient lift should be used solely for transferring a patient from one utility (beds, chairs, toilets, etc.) to another. The patient lift must not be used for transporting or moving any patient from one location to another location.
- During lifting or lowering, always keep the base of the lift in the widest position.
- Do not lock casters during lifting.
- The base of the lift must be closed while moving the lift.
- Do not roll casters over any object.
- Always keep the patient centered over the base and facing the caregiver operating the patient lift.
- Never leave the patient unattended.



## **WARNING!**

- No modification of this equipment is allowed. It is unsafe to modify the equipment from its original state. This product is deemed safe to use with only the original parts as delivered. Unauthorized use of any modified equipment may result in serious injury or death.
- Do not replace any components of the lift without consulting with your dealer, and follow proper instruction from the dealer when replacing any components.
- Do not service any parts of the lift while in use with a patient.
- It is not safe to load the equipment more than the indicated maximum safe load.
- **If** any accessories with lower load capacity than the equipment itself is used, do not load the equipment more than the lowest maximum safe working load.
- Charge the battery at least once a month to maintain its life, or it may deplete completely. Unplug the charger when fully charged to prevent damage.
- Using accessories, detachable parts, or materials not described in the instruction manual may result in serious injuries.
- Keep the equipment away from children and pets. It may result in injuries or death.
- Keep the equipment away from pests or pest infested area. Diseases transferred from pests to the equipment may cause serious illness to patients upon contact.
- If the lift is exposed to the sun for long periods of time, be cautious as the surface of the lift may be hot.
- The controller contains parts and assemblies that could be susceptible to damage by electrostatic discharge (ESD). To reduce the likelihood of this, one should be grounded prior to using the controller.
- Do not use the lift on an inclined surface.
- Do not rock or handle the equipment in side-to-side motion. Excessive sideways load on the actuator may cause it to break and fail which may cause injuries or death.
- A black plastic hardware cap (if applicable) may be a choking hazard. If the cap does not stay on the hardware, replace it immediately.




## **INFORMATION**

- There are no risk if the lift is exposed to reasonably foreseeable environmental conditions.
- This equipment does not consist of any materials or ingredients to which patients are exposed, that may cause bodily harm or injury.
- Prolonged exposure or physical contact with any part of the equipment will not cause skin irritation, allergic reaction, etc.
- The lift will not have potential electromagnetic or other interference with other devices.

# FEATURES & OVERVIEW



- Prior to assembly, unpack and inspect all parts from the shipping carton. Contact your dealer immediately if any parts are damaged or missing.
- Easy tools are provided to assist with initial set up. To ensure maximum safety, product should not be put into use until all connections are tightened using standard tools.





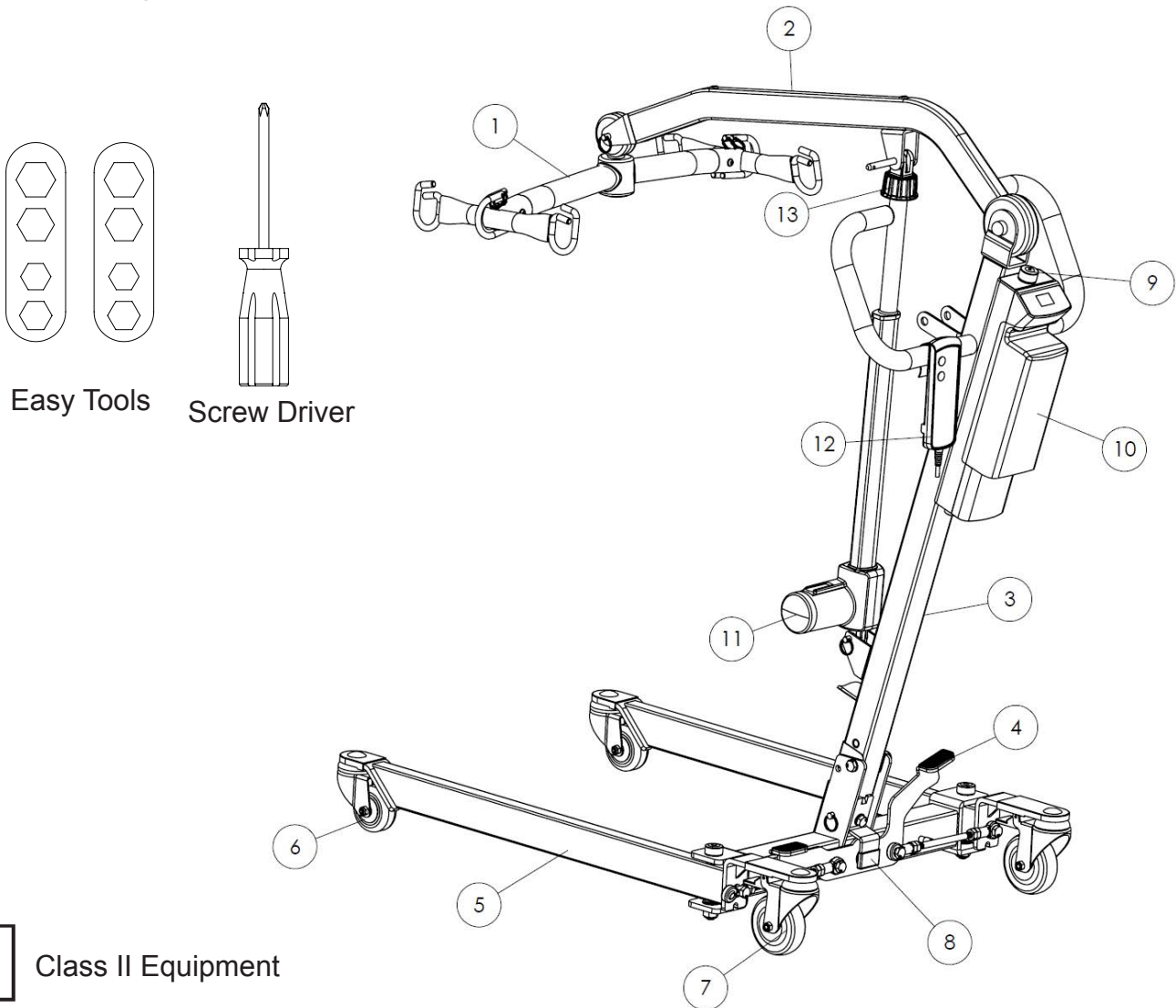


Class II Equipment

1	Spreader Bar	Combined Weight (1-5) 69.8lb (31.7 kg)	6	Front Caster	11	Actuator	7.6 lb (3.5 kg)
2	Boom		7	Rear Caster	12	Hand Control	0.7 lb (0.3 kg)
3	Mast		8	Standing Bracket	13	Emergency Lowering	
4	Leg Opening Pedal		9	Emergency Stop Button		Charger	0.6 lb (0.3 kg)
5	**Base		10	Control Unit			11 lb (5 kg)

\*\* Heaviest detachable part



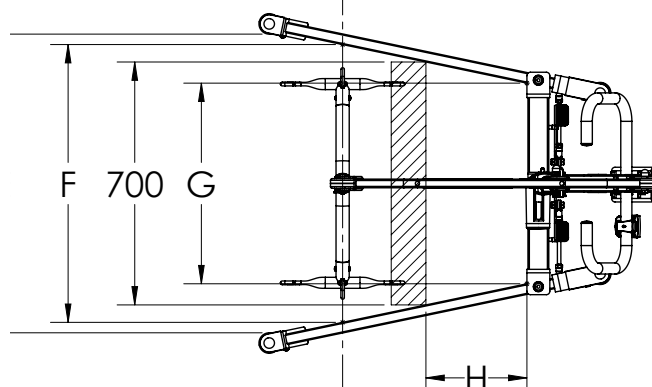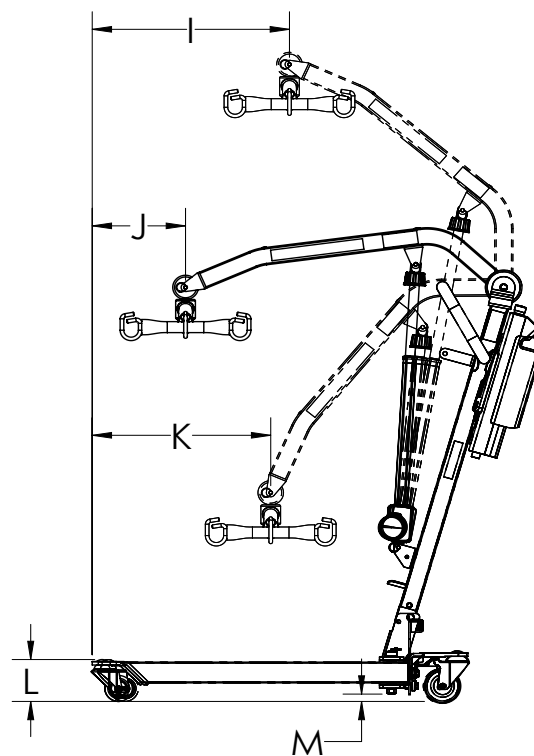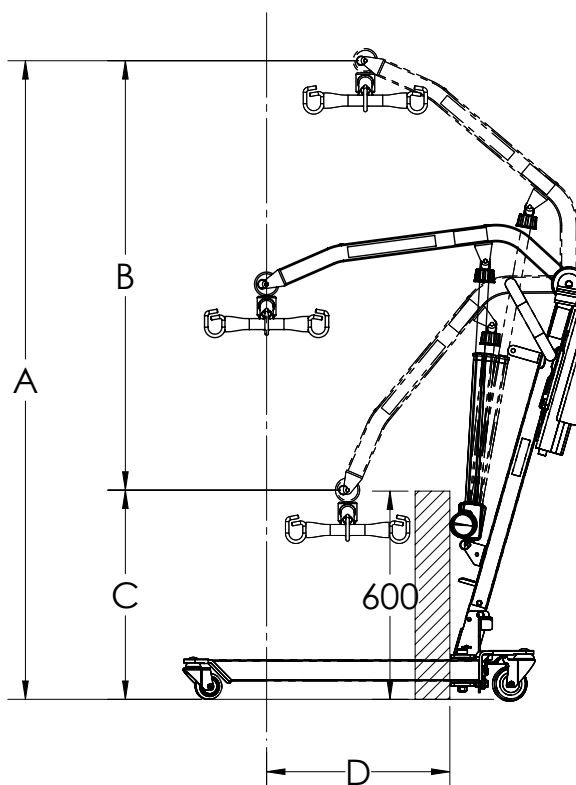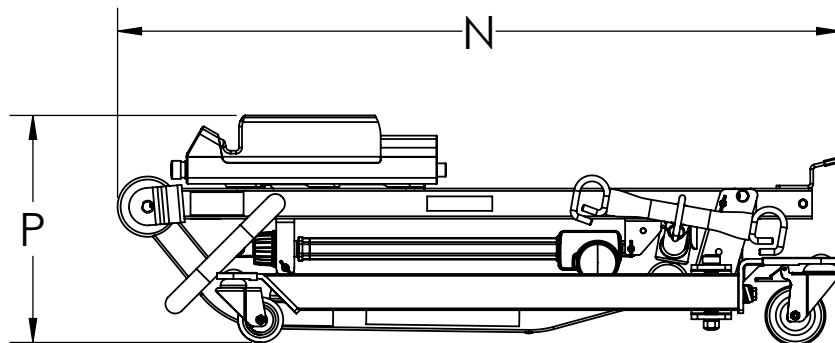
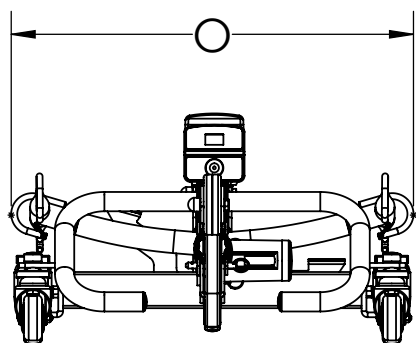


	mm	(in)
A	1841.2	(72.5)
B	1237.5	(48.7)
C	603.7	(23.8)
D	526.7	(20.7)
E	812.8	(32.0)
F	771.4	(30.4)
G	576.6	(22.7)
H	336.4	(13.2)
I	544.1	(21.4)
J	258.4	(10.2)
K	491.0	(19.3)
L	121.0	(4.8)
M	20.3	(0.8)
N	1198.8	(47.2)
O	657.5	(25.9)
P	377.1	(14.8)



# SPECIFICATIONS & OPTIONS

Specifications of Control Unit	
Input Voltage	110-120VAC 60Hz / 220-240VAC 50Hz
Output Voltage	24 VDC
Battery Pack	24 VDC 5AH
Lift Control	Handset and Panel
Protection Class	IP54 (Protection against dust and splash of water)
Sound Level Under Full Load	16db
Sound Level Under No Load	11db
Lift Cycles Per Charge	25 full cycles under full load
Overload Warning	by LCD
Battery Status	by LCD
Duty Cycle	10% Duty cycle (Max 2 minute ON,18 minute OFF)

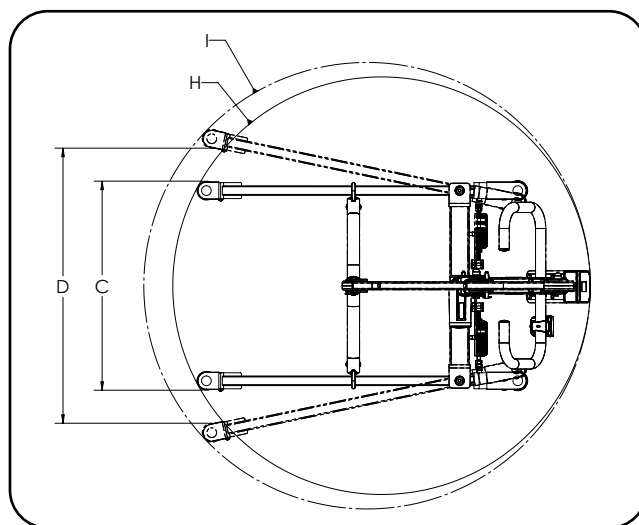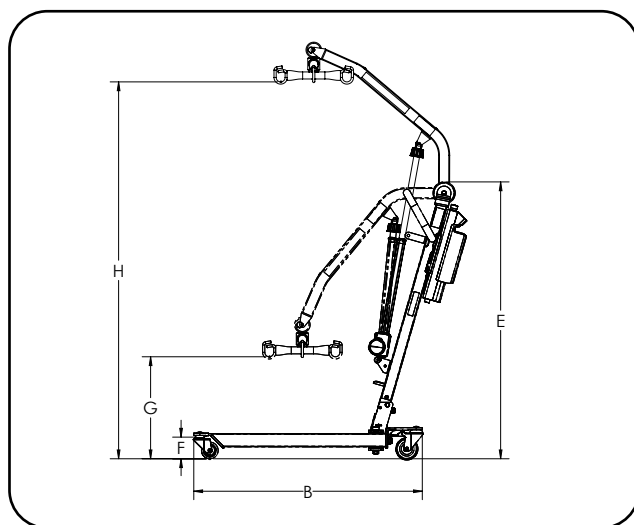


Dimensions & Specifications		
A	Safe Working Load	400 lb 182 kg
B	Base Length	41.0" 1041mm
C	Base External Width – Closed	25.6" 650mm
D	Base Internal Width – Opened	32.0" 813mm
E	Mast Height	48.3" 1227mm
F	Front Caster Height	4.7" 119mm
G	Lifting Height – Lowest	18.5" 465mm
H	Lifting Height – Highest	67.2" 1707mm
I	Base Opened Turning Diameter	55" 1384.9mm
J	Base Closed Turning Diameter	51" 1306.5mm
	Unloaded Lift Weight	89.7 lb 40.8 kg

Standard Features
• 6-Point Spreader Bar
• 4" Rear Caster with Brake
• 3" Front Caster
• Power Control Unit
Optional Features
• Charging Cradle
• Digital Scale
• Additional Battery Pack


\* We are committed to continuous improvements of our products. Therefore, the specification, dimensions, and features listed above are for guidance only and are subject to change without prior notice.

# INSTALLATION GUIDE


**Step 1:** Remove all contents from the carton.

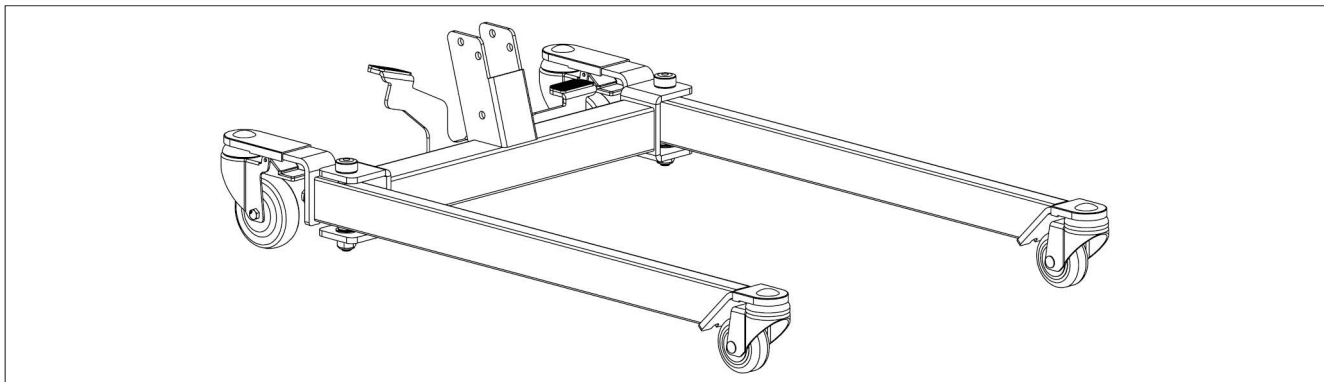


**Step 2:** Install the stand bracket to the mast. Make sure the stand bracket's tab is inserted into the top hole. Use wrenches to tighten the hardware.

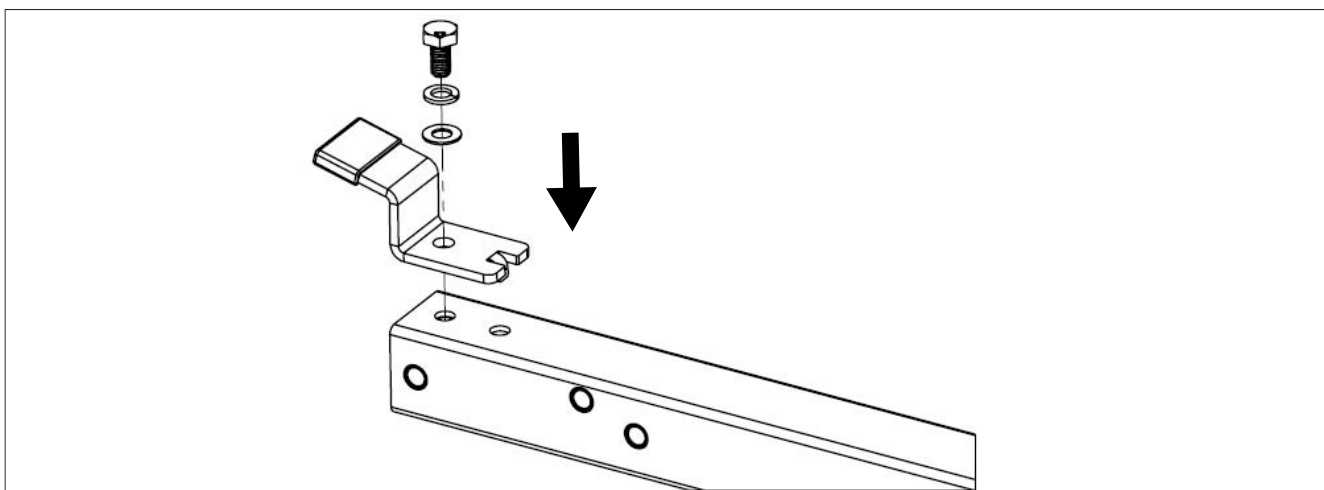


**Step 3:** Lock the rear casters on the base and insert the mast into the base bracket by aligning the holes. Secure the mast to the base using the hardware as shown. Use wrenches to tighten the hardware.

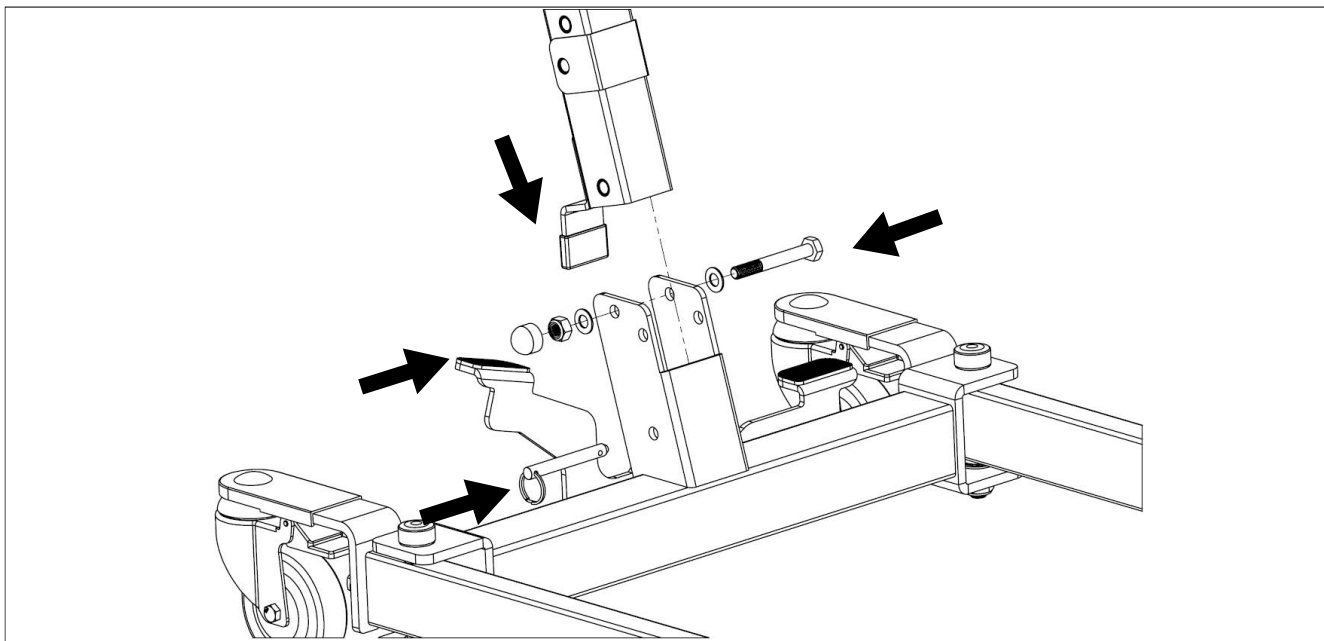


**Step 4:** Attach the joint cover to the boom as shown. Insert the boom to the mast and secure it using the hardware. Use wrenches to tighten the hardware.

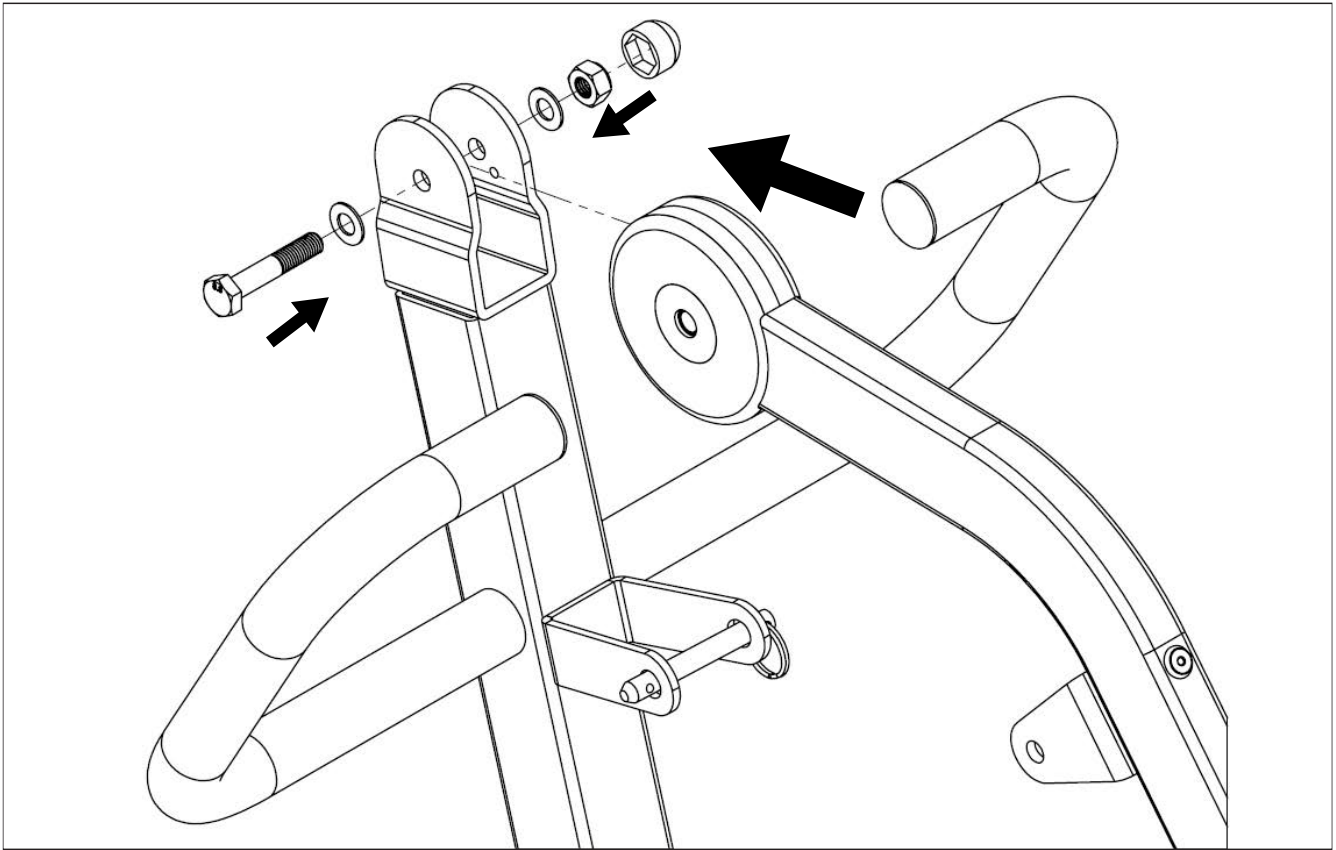


**Step 5:** Move the boom out of the way and install lower end of actuator as shown. It is critical to have motor part of the actuator face the same direction as the big arrow.

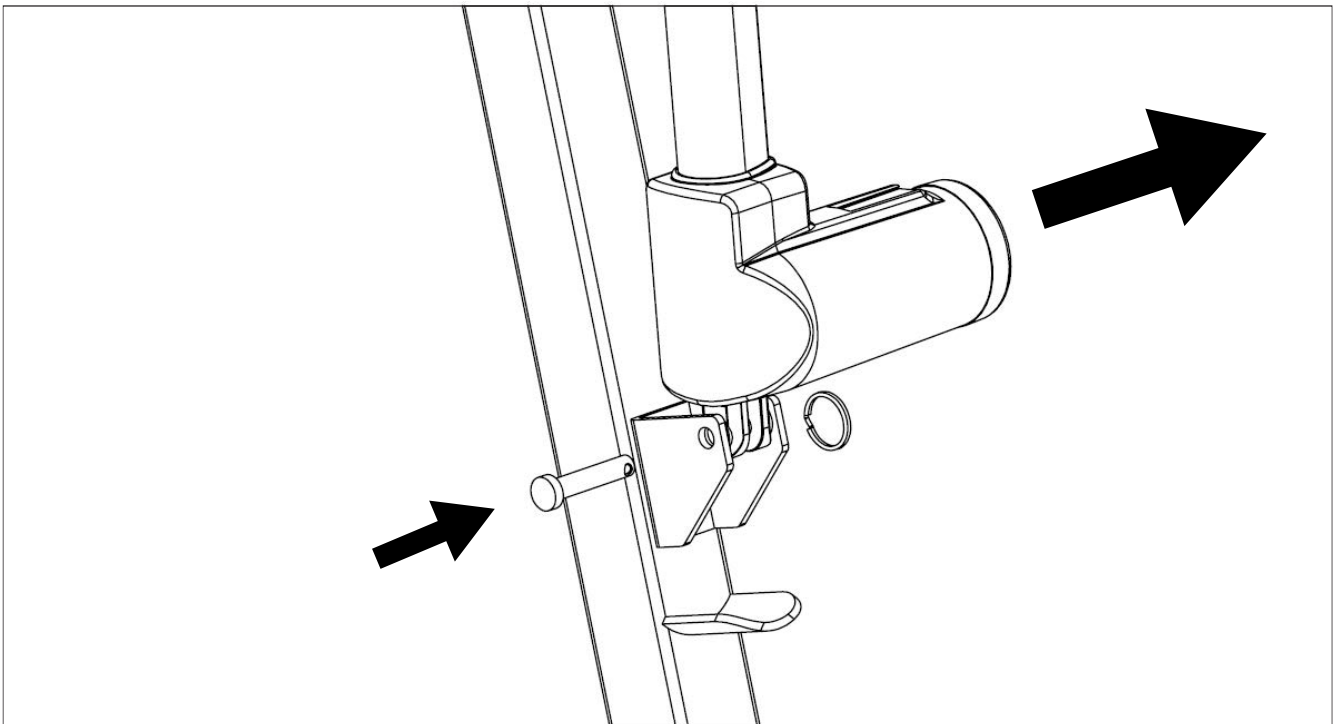


**Step 6:** Install upper end of the actuator by using the quick-release pin.

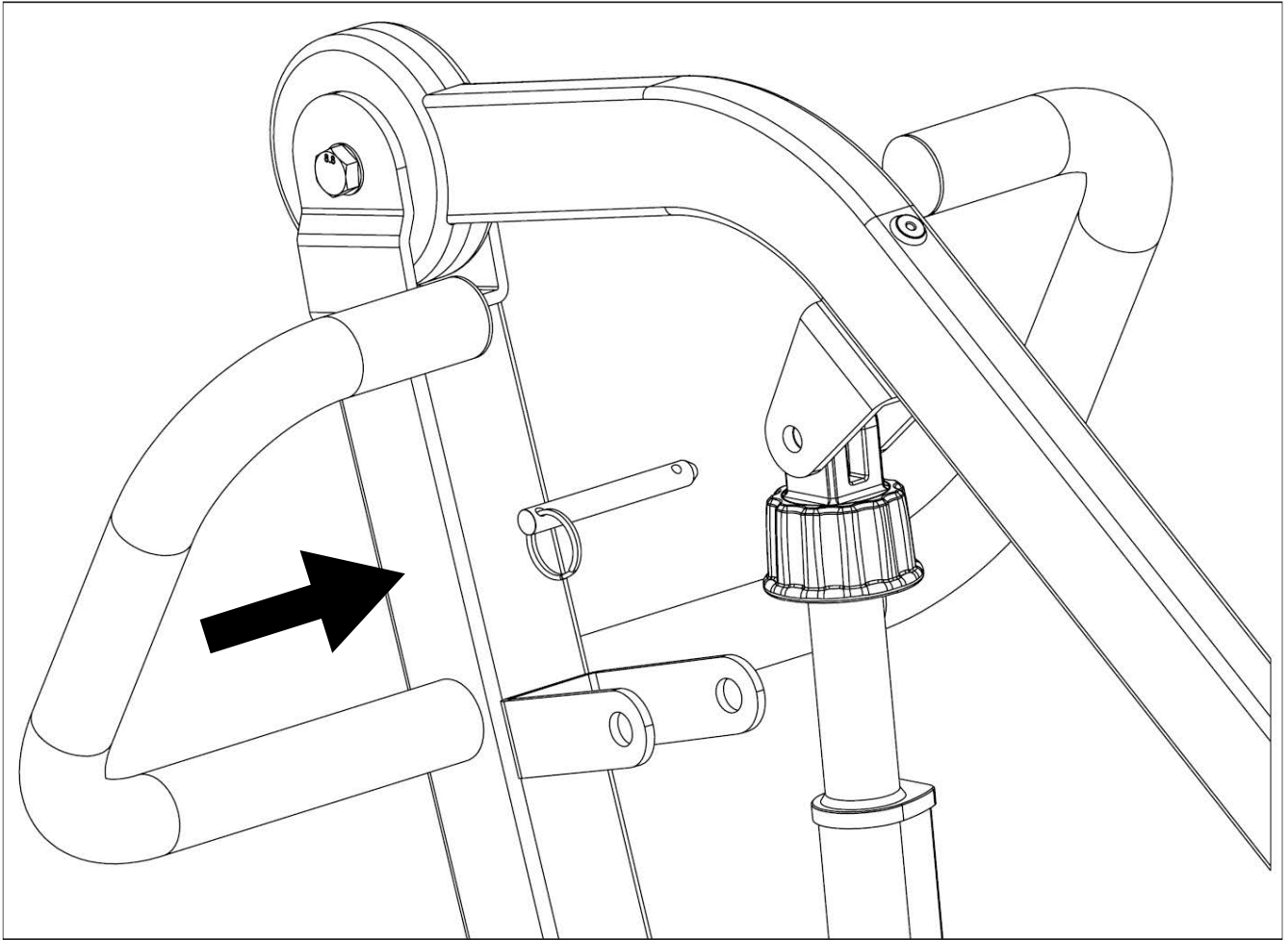

**Step 7:** Install spreader bar into the boom as shown.




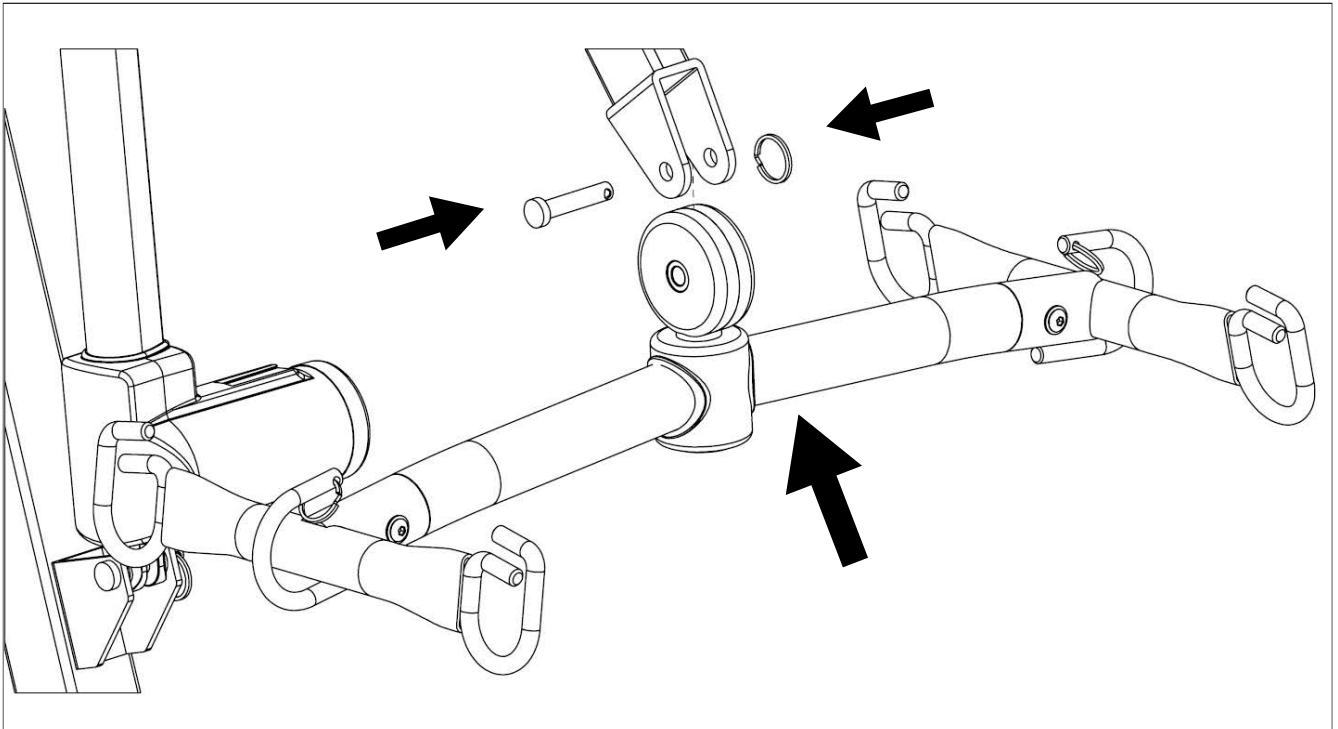


**Step 10:** Install the control unit to the frame as shown.

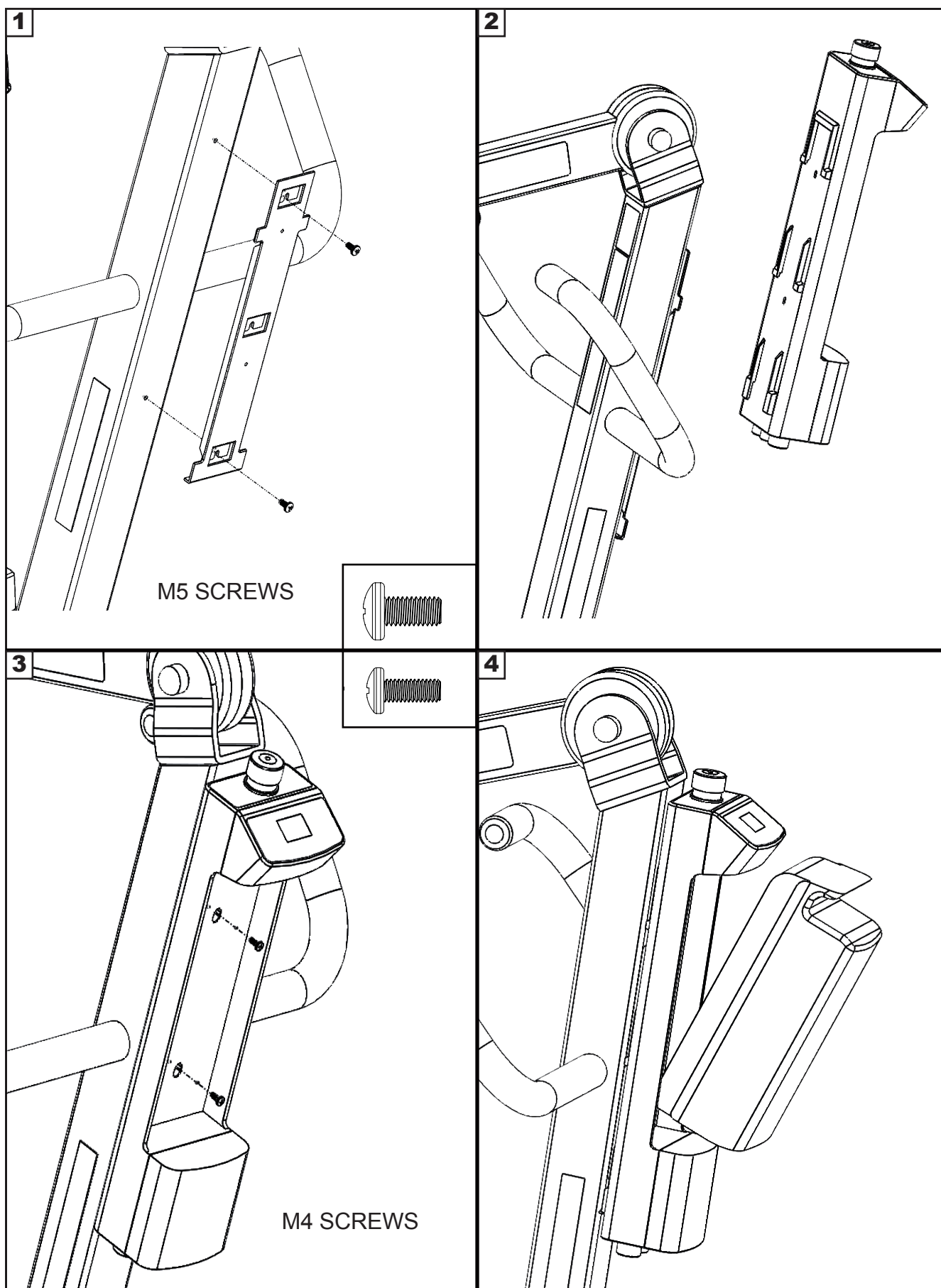




# Folding the PL400EF for Storage or Transport
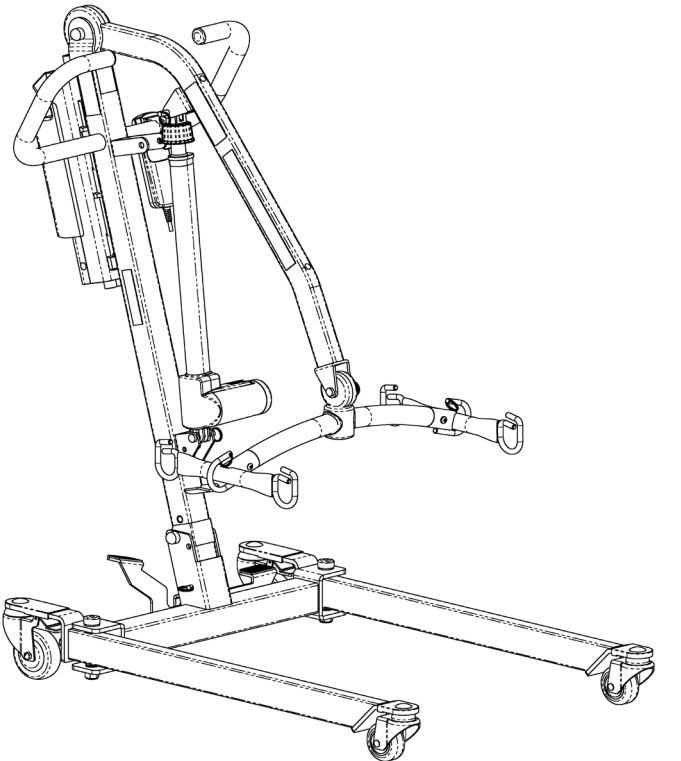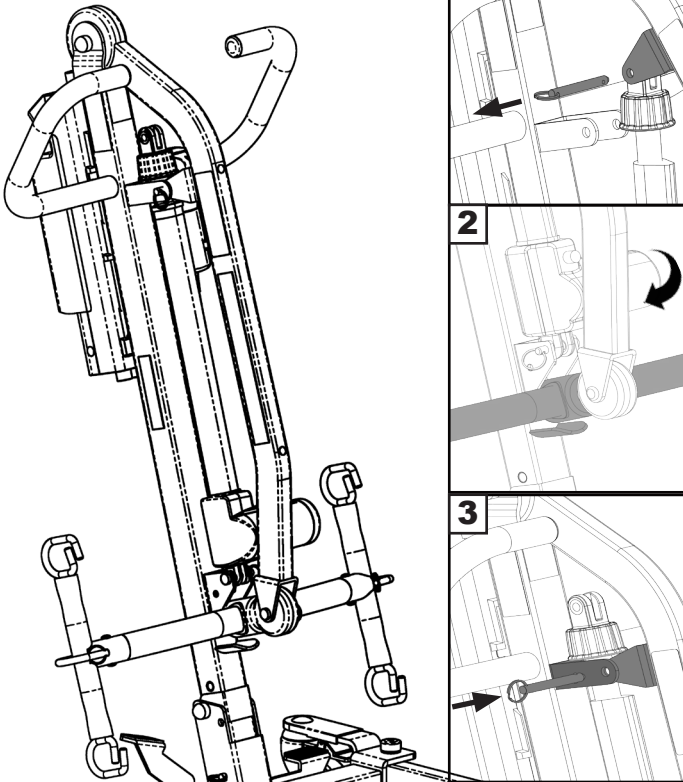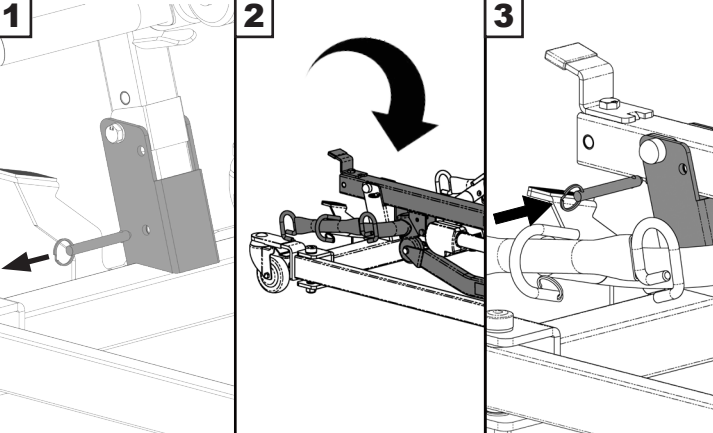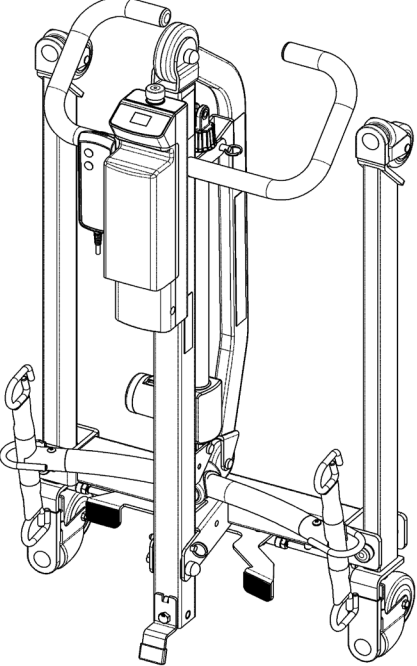
<p><b>Step 1: Spreader bar at lowest position</b></p> 	<p><b>Step 2: Folding the boom</b></p> 
<p><b>Step 3: Folding the mast</b></p> 	<p><b>Step 4: Standing position</b></p>  <p><b>Warning - The lift may fall and cause bodily injury if it is not secured against the wall.</b></p>

# Operating Instructions

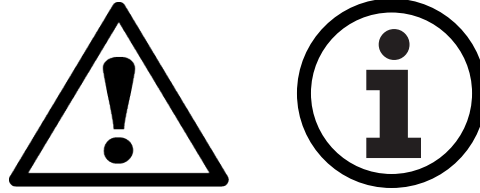

**Double check all assemblies for tightness and read operating instructions carefully prior to use.**

**For optimum performance the lift should be transported and stored in following condition range:**

- -25°C to +5°C (-13°F to 41°F), and
- +5°C to +35°C (41°F to 95°F) at a non-condensing relative humidity 0% to 90%
- +35°C to +70°C at a water vapor pressure up to 5 kPa

**For optimum performance the lift should be operated in following condition range:**

- at a temperature range of +5°C to +40°C (+41°F to +104°F)
- at a non-condensing relative humidity range of 15 % to 90 %, but not requiring a water vapor partial pressure greater than 5 kPa
- 70 kPa to 106 kPa atmospheric pressure



## **PREPARATION BEFORE LIFTING**


- Twist the RED EMERGENCY STOP BUTTON clockwise.
- Verify the Lifting and Lowering Functions by pressing either the UP or DOWN button on the hand control once.
- Position the base of the lift around or under the object.
- Widen the base and disengage the caster brakes.



Do not attempt to transfer a patient without prior approval of the person's nurse. Also, do not transfer without having studied the instructions and performed several practices in operating the product. Together (with the patients doctor, nurse, or medical attendant) select a sling that is both practical and comfortable. The sling selected should be one that serves the needs of the patient, while providing the patient with optimal safety. Never interfere with the lift, unless instructed by the attendant. Have a doctor, nurse, or medical attendant (experienced in the use of the BestStand) present during the first few times the lift is used to transfer a new user.



# Power Control Unit Overview

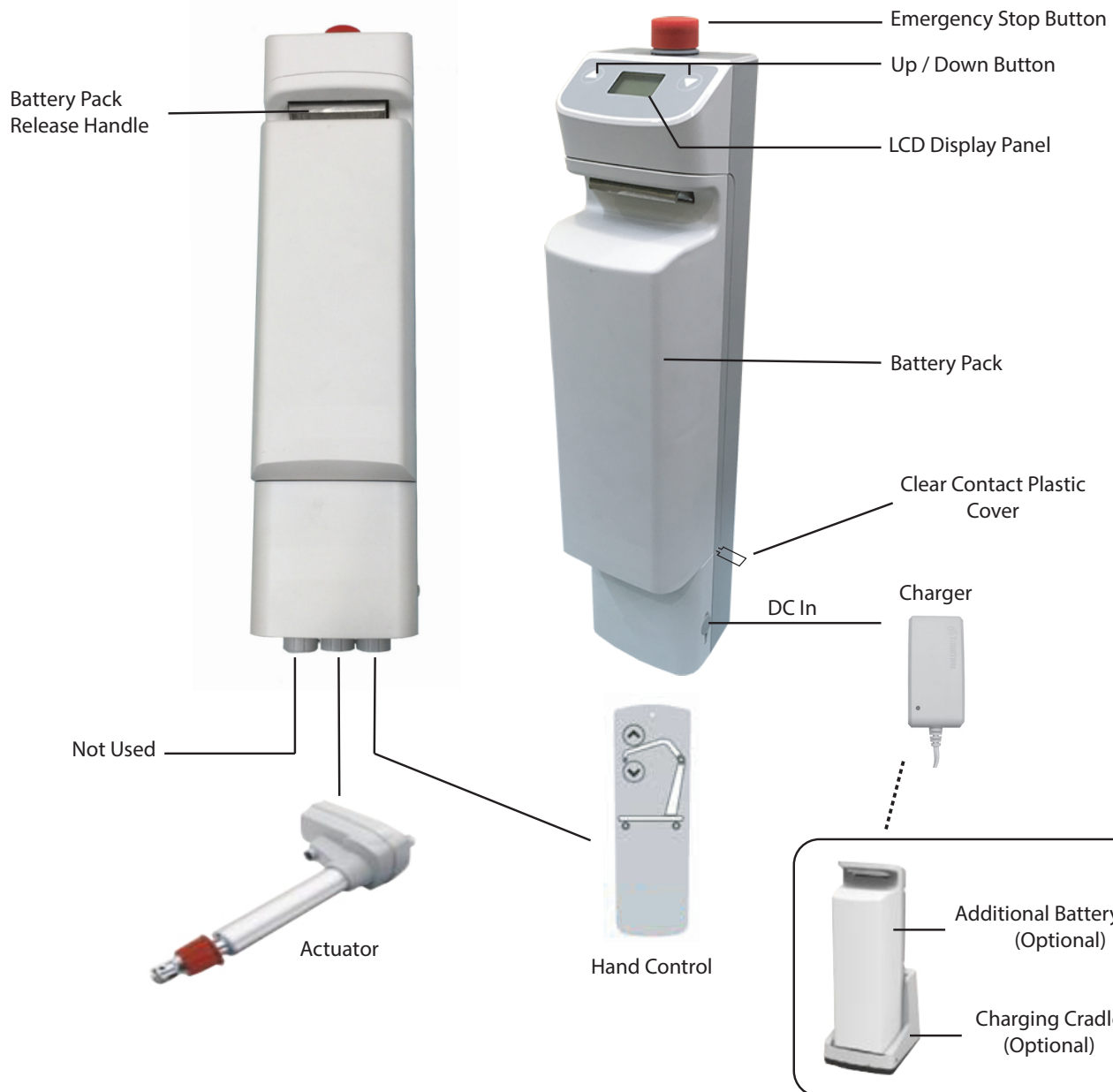

1. Connect the actuator as shown.
2. Connect the hand control as shown.
3. The battery should be charged daily to prolong the battery life.
4. Remove the clear battery contacts plastic cover before use. Remove the battery in order to remove the plastic cover.



## WARNING!


- Contains a long charger cord that may cause strangulation if used improperly.
- Keep long charger cord out of walkway where it poses a tripping hazard.
- When removing the battery, use both hands as it can cause injury if dropped.
- Battery contacts must not touch any objects. Doing so will short the battery and cause fire which may damage the equipment.
- Improperly replacing battery cells may cause damages to both control box and battery beyond repair. Contact the manufacturer before replacing battery cells.




## LCD Display Panel Signs

When the emergency stop button is released, the LCD Display Panel will show one of the four signs below.

- The sign will display for 5 seconds.
- Then the lift will go into standby mode and the sign disappears.
- The lift is ready for use.

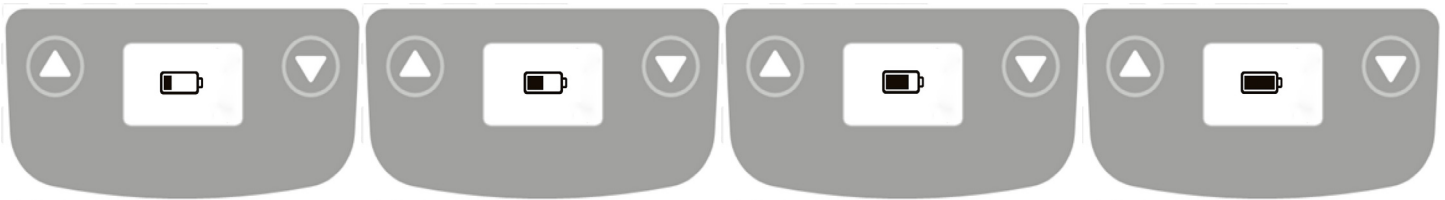


## Charging: DC Charger or Charging Cradle

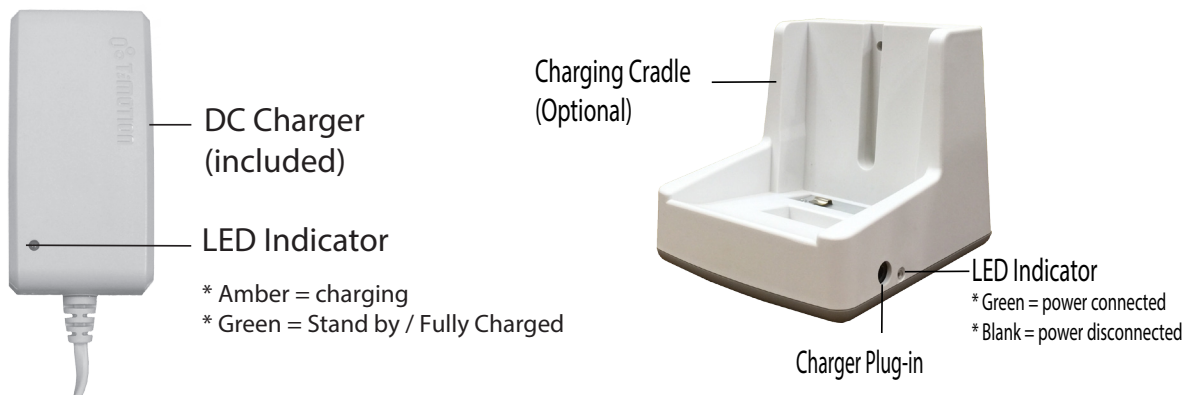


## LCD Display Panel Signs - Charging





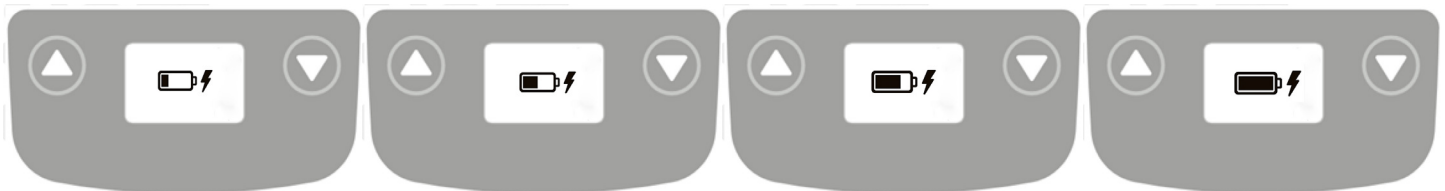

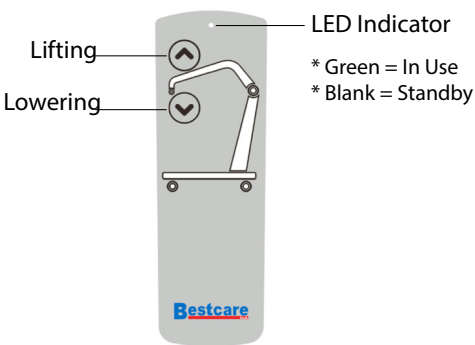

During charging, the LCD Display Panel will show one of the four signs below.

- The DC Charger must be connected.
- The emergency stop button must be released.
- The battery will not be charged if the emergency button is pressed in.










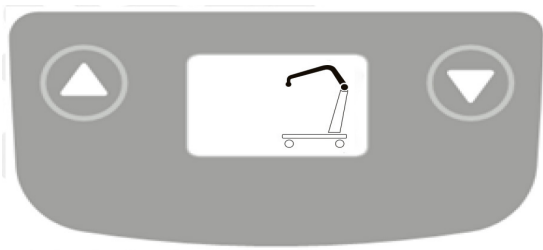

# Operating Lift: Using Hand Control




Button	Press Button	Release Button
	 	
	 	

# Operating Lift: Using LCD Display Panel



Button	Press Button	Release Button
	 	
	 	

## Warning! - Battery Low and Charging is Needed

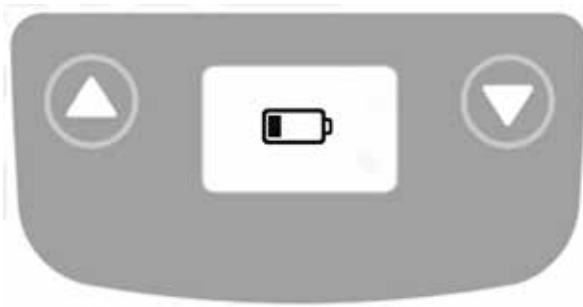

If the battery needs to be charged, the LCD Display Panel will show a blinking low battery sign shown on the left either when the emergency stop button is released or when a button on the LCD Display Panel is pressed.

- Lift is still functioning.
- Buzzer will beep every 2 seconds.
- Battery needs to be charged immediately after use.

## Warning! - Overloaded



During lifting if the lift is overloaded, the LCD Display Panel will show the sign on the left and lifting will be deactivated immediately.

- The buzzer will beep 3 times.
- Lowering the lift by pressing the DOWN button is still allowed.
- The emergency stop button must be pressed in and then release the button before further lifting.

## Warning! - Charge Battery Immediately



If the battery is critically low, the LCD Display Panel will show the sign on the left either when the emergency stop button is released or when a button on the LCD Display Panel is pressed.

- Lift operation is deactivated.
- Battery needs to be charged immediately before further use.

# Available slings compatible with Patient Lift

Sling Type	Material	Sizes
Universal	Mesh	M, L
	Polyester	M, L, XXL
Universal with Head Support	Mesh	S, M, L
	Polyester	S, M, L, XXL
Universal Disposable	Nonwoven, Disposable	M, L
Universal Disposable with Head Support	Nonwoven, Disposable	S, M, L, XXL
Sani	Polyester	S, M, L, XXL
Invacare® Replacement	Polyester	M, L, XL
	Mesh	M, L, XXL
Invacare® Replacement with Head Support	Mesh	M, L, XL, XXL
Hoyer® Replacement	Polyester	M, L, XL
Hoyer® Replacement with Head Support	Polyester	M, L, XL
Liko®/Guldmann® Replacement	Polyester with PU Coating	M, L, XL
Liko®/Guldmann® Replacement with Head Support	Polyester with PU Coating	M, L, XL
Liko®/Guldmann® Disposable Replacement	Nonwoven, Disposable	M, L, XL
Liko®/Guldmann® Disposable Replacement with Head Support	Nonwoven, Disposable	M, L, XL
Pediatric with Head Support	Mesh	Standard
	Polyester	




**If the patient is pregnant or has skin conditions, consult with a doctor before use.**




**ALWAYS check that slings weight capacity is suitable for the patient.**




**All slings are type BF applied parts.**

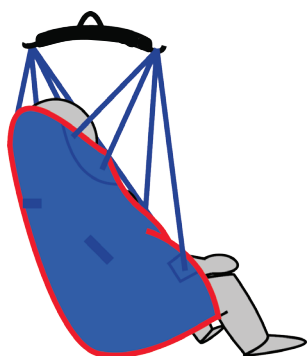


# Sling Hook-Up Guide

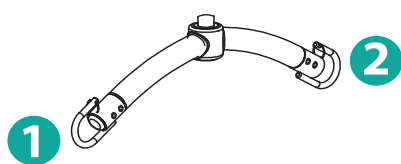

2 Point S-Bar

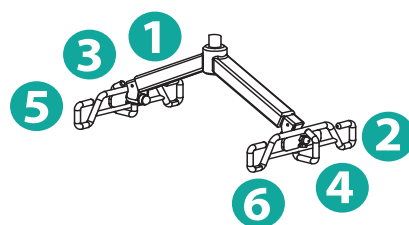


6 Point S-Bar

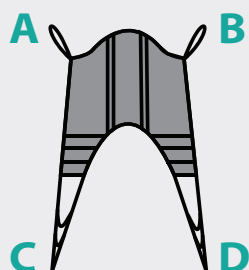

4 Strap Sling

Hook up A ➡ Point 1  
Hook up B ➡ Point 2  
Insert D through C  
Hook up D ➡ Point 1  
Hook up C ➡ Point 2


Hook up A ➡ Point 1  
Hook up B ➡ Point 2  
Insert D through C  
Hook up D ➡ Point 5  
Hook up C ➡ Point 6

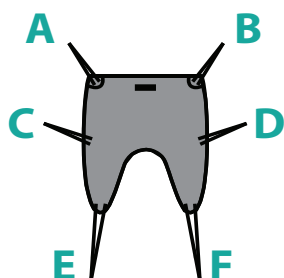

6 Strap Sling

Hook up A, C ➡ Point 1  
Hook up B, D ➡ Point 2  
Insert F through E  
Hook up F ➡ Point 1  
Hook up E ➡ Point 2

Hook up A ➡ Point 1  
Hook up B ➡ Point 2  
Hook up C ➡ Point 3  
Hook up D ➡ Point 4  
Insert F through E  
Hook up F ➡ Point 5  
Hook up E ➡ Point 6

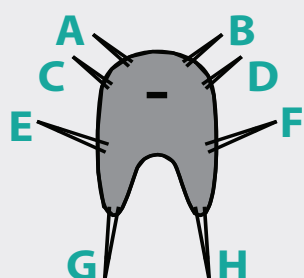

8 Strap Sling

Hook up A, C, E ➡ Point 1  
Hook up B, D, F ➡ Point 2  
Insert H through G  
Hook up H ➡ Point 1  
Hook up G ➡ Point 2


Hook up A & C ➡ Point 1  
Hook up B & D ➡ Point 2  
Hook up E ➡ Point 3  
Hook up F ➡ Point 4  
Insert H through G  
Hook up H ➡ Point 5  
Hook up G ➡ Point 6

## Sling Loop Options

SAFETY LOOP

CENTER LOOP

SHORT LOOP

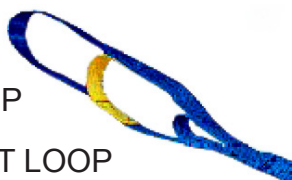

SAFETY LOOP - DO NOT USE FOR LIFTING

\*\*safety loop is used with the center loop or short loop.

CENTER LOOP - reclined position

SHORT LOOP - upright position



**BEFORE LIFTING THE PATIENT, PULL ON EACH SLING LOOPS TO ENSURE THE LOOP IS HOOKED ONTO THE SPREADER BAR PROPERLY.**

## **Transfer From Bed to Wheelchair**

1. User/patient should be in the center of the bed.
2. Position user onto his/her side by rolling user towards you.
3. Roll the sling in half. The handle on the back section should face outward when the sling is fitted.
4. Position the sling under the user/patient so the commode aperture aligns with the base of the spine and top of the sling close to the neck.
5. Roll user onto the opposite side and position him/her on the flat section of the sling. This will allow you to unroll the remainder of the sling from the other side of the user.
6. Once the sling has been positioned centrally, feed the leg sections under the thighs and draw them up between the thighs.
7. Raise the head of the bed if this function is available.
8. Move the lift slowly towards the user and position the spreader bar over the user's/patient's chest. Attach loops of the sling to the hooks on Spreader Bar as shown in the Sling Hook-Up Guide.
9. Lift the user off the bed using the hand control.
10. Pull lift away from bed. Position user over the wheelchair or chair then lower the patient onto the surface.
11. Reverse the above procedures when returning the user/patient to bed.

## **Transfer From Wheelchair to Bed**

1. Grasp the sling at each corner of the "U" shape of the commode aperture.
2. The sling should be fitted with the grab handles on the back-section facing outward.
3. Help the user lean forward slightly, then slide the sling down between the chair and the user's back.
4. Position the commode aperture where the buttocks meets the seat.
5. Position the sling equally around both sides of the body.
6. Draw the leg sections to the front along the length of the user's thigh.
7. Check the sling's central positioning by comparing the lengths of the leg sections when they are drawn forward.
8. Reposition the sling if the leg sections are not equal in length.
9. Feed the leg sections under the thighs.
10. From between the legs, gently pull the leg section up the inner thigh. Feed as much material as possible under and between the thighs. Ensure the leg sections are positioned midway under the thighs to provide good support and greater comfort.
11. Move the lift slowly towards the user and position the spreader bar over the user's chest. Attach loops of the sling to the hooks on Spreader Bar as shown in the Sling Hook-Up Guide.
12. Lift patient above the wheelchair by using the hand control.
13. Pull lift away from wheelchair. Position patient over bed and lower patient onto bed.
14. Reverse the above procedures when returning the user/patient to a wheelchair.



# Maintenance & Inspection



- It is important that certain basic checks are performed periodically by the operator of the lift or by a qualified technician or maintenance staff.
- The manufacturer recommends that the following components and operating points be scheduled for periodic inspection at intervals not greater than 6 months ( $\leq 6$ ). The manufacturer also recommends visual inspection prior to use for the additional points below (V). The periodic inspection interval and the visual inspection points are listed below.
- Note – The manufacturer assumes that the assembly instructions were performed properly and by a qualified individual. The manufacturer also assumes that the operator of the lift has been properly trained, reviewed all the training material and knows how to operate the lift.
- With proper use and care, the expected lifetime of the lift is 10 years or 20,000 cycles.
- The expected lifetime of the electrical components are 3 years.
- Contact the dealer for any recycling information.

	V upon use	$\leq 6$ months
<b>Boom and Spreader Bar</b>		
Check connections between boom and spreader bar for proper connection with pin & key ring (for Patient Lifts)	X	X
Check connection between boom and mast for improper connection, looseness, or wear		X
Check spreader bar hook bolts for looseness or damage (for Patient Lifts)	X	X
Key rings are present on spreader bar hooks (on patient lifts) and on boom hooks (on stand assist lifts)	X	X
<b>Mast</b>		
Check mast for bending or deflection		X
<b>Base and Foot Pedal</b>		
Check all nuts and bolts for looseness		X
Check all casters and axle bolts for tightness		X
Check all casters for rotation and locking/unlocking (rear only)		X
Apply multi-purpose grease to bearings if needed		X
Check all weld joints for rust and cracks		X
<b>Actuator and Control Box</b>		
Check control box is securely affixed to the mast, and battery pack is secured in control box		X
Check actuator is secured between the boom and the mast with the pins and key rings	X	X
Check manual emergency lowering mechanism on actuator operates properly		X
Check red button on control box shuts off all power when pressed		X
Check all cables at the bottom of the control box are properly connected		X
<b>Sling</b>		
Inspect the body of the sling and all loops or connectors that could attach to the spreader bar for signs of wear, damage, or fraying	X	X
<b>Lubrication</b>		
Lubricate all moving joints between hardware and frames to minimize excessive wear and noise		X
<b>Lift Cleaning</b>	As needed	As needed

# **Cleaning and Disinfecting**

## **Cleaning of the lift**

- Use pH-neutral detergents only.
- Remove the battery from the controller.
- Clean lift surfaces, control box and hand control with a damp cloth using pH-neutral detergent.
- Remove debris and hair from the casters and make sure the casters can spin and swivel freely.



- Solvents or strong liquids or abrasive cleaning products will damage the surfaces of the lift.

## **Cleaning of the sling**

- Check with sling label for washing condition.
- As for minor cleaning between patient uses, we recommend wipes with water-soluble soap or isopropyl alcohol.
- For slings with rigid supports, remove the plastic inserts before washing. (Look for an opening near the plastic inserts for removal).
- Machine wash warm or cold.
- Do not wash with other colors.
- Air dry or very low dryer heat.



- DO NOT USE BLEACH.
- DO NOT clean or wash disposable slings.

## **Deterioration of sling**

- Contact the dealer for “Guideline for Identifying Deteriorated Slings” if needed.
- Replace a sling when it shows signs of deterioration.

## **Disinfecting of the lift and sling**

- Inspect slings prior to each use for contamination from previous use.
- Use isopropyl alcohol for any disinfection purpose.



- Wash a reusable sling for serious contamination.
- Replace a disposable sling for any contamination.
- Reusing a contaminated sling can cause serious bacterial infection.

# **Troubleshooting Guide**

The following list of encountered problems and solutions will assist you in determining what may be causing your patient lift not to function as designed. If you have a problem occurring which is not listed below please contact your dealer or technical support for help. Do not attempt to repair or replace components or parts on your lift as this may void your warranty or cause further problems that may result in patient injury. Stop using your lift immediately if it is not functioning correctly or any warning beeps are heard.

***I need to re-charge my batteries often or they fail to hold a charge when I charge them.***

- Replace the batteries or battery pack as they are at the end of their life cycle
- Batteries should be changed every 18-24 months depending on usage

***The actuator will either go up or down but not both***

- Replace your hand control as it is at the end of its life cycle
- Your control box may be malfunctioning

***There is a grinding sound inside the actuator when lifting***

- Replace your actuator as the internal gears are stripped

***The actuator stops and starts while lifting or lowering***

- Replace the batteries or battery pack as they are at the end of their life cycle

***My lift will not operate even though it shows the batteries are charged***

- Unplug the AC power cord from the control box as the lift will not work when the control box/charger is connected to AC Power.
- Check to make sure the hand control is properly inserted all the way into the control box port.
- Check to make sure the actuator plug is properly connected and inserted all the way into the control box
- Your hand control needs to be replaced as it is at the end of its life cycle

***If there is a problem you could not solve, please contact your dealer for assistance.***




# **Emergency Lowering Mechanism**

**Contact your dealer immediately if standard troubleshooting techniques do not correct the failure. Do not attempt to lift until all failure and safety issues have been resolved.**

In case of lift failure, please follow the procedures below to safely lower the user.

The Emergency Lowering Device is located at the top of the actuator shaft. It is intended for use if the actuator or electronics fail to operate while the patient is suspended in midair. The device consists of a plastic collar ring that should be first pulled down and turned clockwise continually until the patient has been lowered.

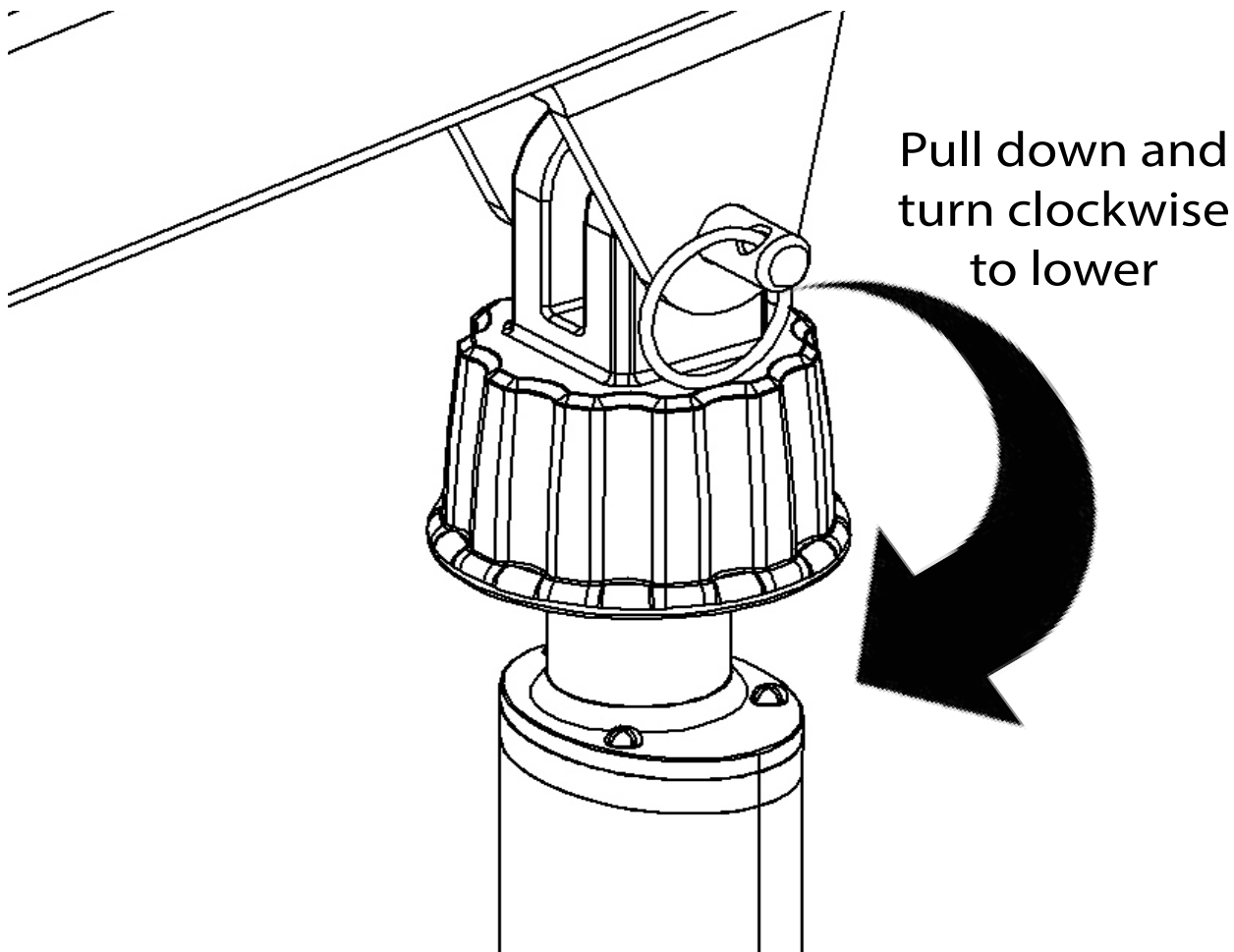

**NOTE:**  
**THE EMERGENCY LOWERING DEVICE IS INTENDED FOR USE DURING LIFT FAILURE. THIS DEVICE WILL ALLOW LOWERING OF PATIENTS ONLY. PLEASE CONTACT YOUR DEALER IMMEDIATELY IN CASE OF FAILURE.**

# **WARRANTY POLICY**

Bestcare LLC offers a limited warranty on all patient lifts, slings and accessories to be free of defects in workmanship and product performance. This warranty is extended only to the original purchaser/user and is therefore non transferable. Any warranty claims must be submitted through an authorized dealer or distributor that sold the original product with proper proof of sale and serial number supplied. Prior authorization from Bestcare LLC is required for any warranty replacement and we reserve the right to replace parts and or accessories in lieu of sending complete new patient lifts. Dealers and distributors will be billed for any advance warranty replacement items shipped and credited at 100% only when the defective items have been returned to Bestcare LLC and proven defective. Product returned without a Bestcare LLC return authorization will be refused. Returns received 30 days after the issuance of a return authorization will also be refused as the authorization will have expired. The final disposition of warranty claims will be subject to our inspection of the returned items. Bestcare LLC has the right to determine valid claims based on the individual circumstances of each claim. Warranty claims will be denied should evidence of the following be determined:

- Misuse of product , modification of product or failure to maintain as per the owner's manual
- Serial number removed or defaced
- Normal wear and tear

Bestcare LLC warrants the following products and components for the time period specified:

- Lift frames and spreader bar: 3 years
- Actuator: 2 years
- Parts including casters, control box, PCB, pendant, charger: 2 years
- Batteries: 1 year
- Weigh Scale: 2 years
- Spryte stand aid all components: 2 years
- Hydraulic pumps: 1 year
- Reusable slings: 6 months

Bestcare LLC Single Patient Slings are designed for limited use with one patient and may not be washed. Bestcare will replace any disposable slings found to have a manufacturing defect. Normal wear and tear will not be covered under warranty.

# NOTES



Bestcare LLC  
2920 Pacific Drive  
Norcross, GA 30071

Rev. 01

[www.bestcarellc.com](http://www.bestcarellc.com) | © 2022 Bestcare LLC